



Yes, You Can.®



*Invacare*® **Leo**











---

**Scooter  
Operating manual**



## How can you get in touch with Invacare®?

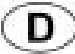If you have any questions or need support, please contact your authorised Invacare® Dealer, who has the necessary know-how and equipment plus the special knowledge concerning your Invacare® product, and can offer you all-round satisfactory service. Should you wish to contact Invacare® directly, you can reach us in Europe at the following addresses and phone numbers.

 <b>A</b>	<b>Mobitec Mobilitätshilfen GmbH</b> Herzog Odilostrasse 101 A-5310 Mondsee <b>Austria</b>	 Fax: @: @: WWW:	+43 - 6232 - 55 35 0 +43 - 6232 - 55 35 4 office@mobitec-austria.com austria@invacare.com www.mobitec-austria.com
 <b>B</b>  <b>L</b>	<b>Invacare® n.v.</b> Autobaan 22 B-8210 Loppem (Brugge) <b>Belgium</b>	 Fax: @: WWW:	+32 - (0)50 - 83 10 10 +32 - (0)50 - 83 10 11 belgium@invacare.com www.invacare.be
 <b>CH</b>	<b>Mobitec Rehab AG</b> Benkenstraße 260 CH-4108 Witterswil <b>Switzerland</b>	 Fax: @: @: WWW:	+41 - (0)61 - 48 77 08 0 +41 - (0)61 - 48 77 08 1 office@mobitec-rehab.ch switzerland@invacare.com www.mobitec-rehab.ch
 <b>D</b>	<b>Invacare Aquatec</b> Alemannenstraße 10 88316 Isny <b>Deutschland</b>	 Fax: @: WWW:	0 75 62 / 7 00 - 251 08 00 / 6 73 81 72 info@invacare-aquatec.de www.invacare-aquatec.de
 <b>DK</b>	<b>Invacare® A/S</b> Sdr. Ringvej 39 DK-2605 Brøndby <b>Danmark</b>	 Fax (Kundeservice): @: WWW:	+45 - (0)36 - 90 00 00 +45 - (0)36 - 90 00 01 denmark@invacare.com www.invacare.dk

**E** **Invacare® SA**  
c/ Areny, s/n  
Poligon Industrial de Celrà  
17460 Celrà (Girona)  
**ESPAÑA**

☎ : +34 - (0)972 - 49 32 00  
Fax: +34 - (0)972 - 49 32 20  
@: contactsp@invacare.com  
WWW: www.invacare.es

---

**F** **Invacare® Poirier SAS**  
Route de St Roch  
F-37230 Fondettes  
**France**

☎ : +33 - (0)247 - 62 64 66  
Fax : +33 - (0)247 - 42 12 24  
@: contactfr@invacare.com  
WWW: www.invacare.fr

---

**GB** **Invacare® Ltd**  
South Road  
Bridgend Industrial Estate  
Mid Glamorgan - CF31-3PY  
**United Kingdom**

☎ (Customer Service): +44 - (0)1656 - 664 321  
Fax (Customer Service): +44 - (0)1656 - 667 532  
@: uk@invacare.com  
@: eire@invacare.com  
WWW: www.invacare.co.uk

---

**I** **Invacare Mecc San s.r.l.**  
Via Dei Pini, 62  
I - 36016 Thiene (VI)  
**ITALIA**

☎ : +39 - 0445 - 38 00 59  
Fax: +39 - 0445 - 38 00 34  
@: italia@invacare.com  
WWW: www.invacare.it

---

**IE** **Invacare Ireland**  
Unit 5 Seatown Business Campus  
Seatown Rd, Swords  
Dublin  
**Ireland**

☎ : +353 - 18 10 70 84  
Fax: +353 - 18 10 70 85  
@: eire@invacare.com

---

**N** **Invacare® AS**  
Grensesvingen 9  
Postboks 6230  
N-0603 Oslo  
**Norge**

☎ (Kundeservice): +47 - (0)22 57 95 10  
Fax (Kundeservice): +47 - (0)22 57 95 01  
@: norway@invacare.com  
WWW: www.invacare.no



**Invacare® B.V.**  
Celsiusstraat 46  
NL-6716 BZ Ede  
**The Netherlands**

☎ : +31 - (0)318 - 69 57 57  
Fax: +31 - (0)318 - 69 57 58  
@: csede@invacare.com  
WWW: www.invacare.nl



**Invacare® PORTUGAL Lda**  
Rua Senhora de Campanhã 105  
P-4369-001 Porto  
**PORTUGAL**

☎ : +351-225105946  
Fax: +351-225105739  
@: portugal@invacare.com  
WWW: www.invacare.pt



**Återförsäljare:**  
**Invacare® AB**  
Fagerstagatan 9  
S-163 91 Spånga  
**Sverige**

☎ (Kundtjänst): +46 - (0) 8 761 70 90  
Fax (Kundtjänst): +46 - (0) 8 761 81 08  
@: sweden@invacare.com  
@: finland@invacare.com  
WWW: www.invacare.se



**Tillverkare:**  
**Invacare® Deutschland GmbH**  
Kleiststraße 49  
D-32457 Porta Westfalica  
**Deutschland**

**MÖLNDAL**  
☎ +46 - (0) 31 – 86 36 00  
Fax: +46 - (0) 31 – 86 36 06  
@: ginvacare@invacare.com

**LANDSKRONA**  
☎ +46 - (0) 418 – 285 40  
Fax: +46 - (0) 418 – 180 89  
@: linvacare@invacare.com

**OSKARSHAMN**  
☎ +46 - (0) 491 – 101 40  
Fax: +46 - (0) 491 – 101 80  
@: oinvacare@invacare.com

# Table of Contents

Chapter		Page
<b>1</b>	<b>Introduction</b>	<b>8</b>
1.1	Guarantee .....	9
1.2	Important symbols in this manual .....	10
1.3	Important symbols found on the vehicle .....	11
1.4	Type classification and permissible use.....	12
1.5	Life expectancy.....	12
<b>2</b>	<b>Safety Notes</b>	<b>13</b>
2.1	General Safety Notes .....	13
2.2	Safety information with regard to care and maintenance .....	16
2.3	Safety Information on Electromagnetic Interference.....	17
2.4	Safety Information on Driving and Freewheel Mode.....	18
<b>3</b>	<b>Key features</b>	<b>20</b>
<b>4</b>	<b>Driving</b>	<b>21</b>
4.1	Before driving for the first time.....	21
4.2	Taking Obstacles .....	22
4.3	Driving up and down gradients.....	23
4.4	Parking and stationary .....	23
<b>5</b>	<b>Pushing the scooter by hand</b>	<b>24</b>
5.1	Disengaging Motors .....	24
<b>6</b>	<b>Operating console</b>	<b>26</b>

6.1	<b>Operating console arrangement</b> .....	<b>26</b>
6.1.1	Status display .....	27
6.1.2	Battery charge display .....	27
6.2	<b>Driving the Scooter</b> .....	<b>28</b>
6.3	<b>Activating and deactivating acoustic signals</b> .....	<b>29</b>
6.4	<b>Diagnosis and fault repair</b> .....	<b>32</b>
6.4.1	Error diagnosis.....	33
6.5	<b>Error Codes and Diagnostic Codes</b> .....	<b>34</b>
<b>7</b>	<b>Adjustment facilities</b> .....	<b>37</b>
7.1	<b>Moving the seat position forwards or backwards</b> .....	<b>37</b>
7.2	<b>Adjusting the armrest width</b> .....	<b>38</b>
7.3	<b>Disengaging the seat to rotate it or remove it</b> .....	<b>39</b>
7.4	<b>Adjusting the seat height</b> .....	<b>40</b>
<b>8</b>	<b>Electrical System</b> .....	<b>42</b>
8.1	<b>Electronics Protection System</b> .....	<b>42</b>
8.1.1	The main fuse .....	43
8.2	<b>Batteries</b> .....	<b>43</b>
8.2.1	What you need to know about batteries .....	43
8.2.2	Charging the batteries .....	45
8.2.3	Removing and fitting batteries .....	47
8.2.3.1	Removing the batteries.....	48
8.2.3.2	How to handle damaged batteries correctly .....	50
<b>9</b>	<b>Care and maintenance</b> .....	<b>51</b>
<b>10</b>	<b>Repair Instructions</b> .....	<b>53</b>
10.1	<b>Repairing a flat tyre</b> .....	<b>53</b>
10.1.1	Mending punctures (pneumatic tyres of type 3.00 - 4) .....	54
10.1.1.1	Repairing punctured tyres .....	55

<b>11</b>	<b>Dismantling the Scooter for transport</b>	<b>57</b>
11.1	Removing the seat.....	57
11.2	Removing the battery and drive unit .....	58
11.2.1	Removing the batteries.....	58
11.2.2	Removing the drive unit.....	58
<b>12</b>	<b>Disposal</b>	<b>60</b>
<b>13</b>	<b>Technical Specifications</b>	<b>61</b>
<b>14</b>	<b>Inspections Performed</b>	<b>64</b>

# 1 Introduction

**Dear user,**

First we would like to thank you for purchasing our product! We hope you will enjoy your new Scooter.

**This manual contains important hints and information on:**

- **Safety**
- **Operation**
- **Care and maintenance.**

**Please take care to read the operating manual thoroughly before starting out on your first journey.**

This product has been designed to fit the needs of different types of users with different requirements.

The decision whether the model is suitable for the user may only be taken by medical specialists with appropriate aptitude.

Invacare® or their statutory representatives can accept no liability in cases in which the mobility product has not been adapted to suit the user's handicaps.

Some maintenance and settings can be carried out by the user. Certain adjustments do however require technical training and may only be carried out by your Invacare® specialist dealer. Damages and errors caused by nonobservance of the operating manual or as a result of incorrect maintenance are excluded from all guarantees.



This manual contains copyrighted information. It may not be reproduced or copied in whole or in part without the prior written consent of Invacare® or its authorised representative. It may also contain information that pertains to models sold only in certain countries. In this case the information will be clearly marked as pertaining to a particular country-specific version. We reserve the right to make any alterations on the grounds of technical improvements.

## **1.1 Guarantee**

The terms and conditions of the guarantee are part of the general terms and conditions particular to the individual countries in which this product is sold.

## 1.2 Important symbols in this manual

---



### **WARNING!**

This symbol warns you of danger!

- *Always follow the instructions to avoid injury to the user or damage to the product!*
- 



### **EXPLOSION HAZARD!**

This symbol warns you of an explosion hazard, which can be caused by excessive tyre pressure in a pneumatic tyre!

- *Always follow the instructions to avoid injury to the user or damage to the product!*
- 



### **BURN HAZARD!**

This symbol warns you of burns due, for example, to leaking battery acid!

- *Always follow the instructions to avoid injury to the user or damage to the product!*
- 



### **NOTE:**

This symbol identifies general information which is intended to simplify working with your product and which refers to special functions.

---



### **Requirements:**

- This symbol identifies a list of various tools, components and items which you will need in order to carry out certain work.
-

### 1.3 Important symbols found on the vehicle



---

This product has been supplied from an environmentally aware manufacturer that complies with the Waste Electrical and Electronic Equipment (WEEE) Directive 2002/96/CE. This product may contain substances that could be harmful to the environment if disposed of in places (landfills) that are not appropriate according to legislation.

- *The 'crossed out wheelie bin' symbol is placed on this product to encourage you to recycle wherever possible.*
  - *Please be environmentally responsible and recycle this product through your recycling facility at its end of life.*
-

## 1.4 Type classification and permissible use

This vehicle was designed for persons whose ability to walk is impaired, but who are still physically and mentally able to operate an electric vehicle. It has been classified according to EN 12184 as a **class B mobility product** (for indoor and outdoor areas). It is therefore compact and agile enough for indoor areas, but also able to overcome many obstacles in outdoor areas.

You can find exact information on speed, turning radius, range, safe climbing ability, maximum obstacle height and permissible operating conditions in chapter "**Technical Specifications**" on page **61**.

Please also pay attention to all safety information in chapter "**Safety Notes**" starting from page **13**.

The vehicle was successfully tested according to German and international standards as to its safety. It was also tested successfully according to EN60529 IPX4 as to its resistance to spray water, and is therefore well suited for typical middle European weather conditions. When equipped with an appropriate lighting system, the vehicle is suitable for use on public roads.

## 1.5 Life expectancy

We estimate a life expectancy of five years for this product, provided it is used in strict accordance with the intended use as set out in this document and all maintenance and service requirements are met. The estimated life expectancy can be exceeded if the product is carefully used and properly maintained, and provided technical and scientific advances do not result in technical limitations. The life expectancy can also be considerably reduced by extreme or incorrect usage. The fact that we estimate a life expectancy for this product does not constitute an additional warranty.

## 2 Safety Notes

- READ WELL BEFORE OPERATION!

### 2.1 General Safety Notes



---

**Danger of injury if this scooter is used in any other way than the purpose described in this manual!**

- *Adhere strictly to the instructions in this user manual!*

**Danger of injury if the scooter is driven when your ability to drive is impaired by medication or alcohol!**

- *Never drive any vehicle under the influence of medication or alcohol!*

**Danger of damage or injury if the scooter is accidentally set into motion!**

- *Switch the power system off before you get in, get out or handle awkward objects!*
  - *Be aware that there are only the motor brakes to stop your scooter. When the motors are disengaged, these brakes are automatically deactivated. For this reason, freewheel operation is only recommended on flat surfaces, never on gradients. Never leave your vehicle on a gradient with its motors disengaged. Always re-engage the motors immediately after pushing the vehicle.*
-



---

**Danger of injury if the On/Off Button is pressed while the vehicle is in motion, due to it coming to an abrupt, sharp stop!**

- *If you have to brake in an emergency, simply release the drive lever, which will bring you to a halt! Only switch the vehicle off while in motion as a last resort!*

**Danger of injury if the scooter is transported in another vehicle with the occupant seated in it!**

- *Never transport the scooter with the occupant seated in it!*

**Danger of injury if maximum permissible load is exceeded!**

- *Do not exceed the maximum permissible load (see technical specifications)!*

**Danger of injury when lifting heavy components!**

- *When maintaining, servicing or lifting any part of your scooter, take into account the weight of the individual components, especially the batteries! Be sure at all times to adopt the correct lifting posture and ask for assistance if necessary!*

**Danger of injury if you fall off the scooter!**

- *If restraining systems are installed (such as seat belts), use them each time you drive the scooter.*
-



---

**Danger of injury by moving parts!**

- *Make sure that no injury is incurred by moving parts of the scooter, like wheels or a Seat Lifter, especially when children are around!*

**Danger of fire or breaking down due to electric devices being connected!**

- *Do not connect any electric devices to your vehicle that are not expressly certified by Invacare® for this purpose! Have all electrical installations done by your authorised Invacare® Dealer!*

**Danger of technical failure and injury if unauthorised spare parts and components are used!**

- *Only use original Invacare® spare parts, which have been approved for use with this vehicle!*
-

## 2.2 Safety information with regard to care and maintenance



---

### **Danger of accident and loss of guarantee if maintenance is insufficient!**

- *For reasons of safety and in order to avoid accidents which result from unnoticed wear, it is important that this electric vehicle undergoes an inspection once every year under normal operating conditions (see inspection plan contained in service instructions)!*
  - *Under difficult operating conditions such as daily travel on steep slopes, or in the case of use in medical care cases with frequently changing wheelchair users, it would be expedient to carry out intermediate checks on the brakes, accessories and running gear!*
  - *If the vehicle is to be operated on public roads, the vehicle driver is responsible for ensuring that the vehicle is in an operationally reliable condition! Inadequate or neglected care and maintenance of the vehicle will result in a limitation of the manufacturer's liability!*
-



## 2.3 Safety Information on Electromagnetic Interference

This electric vehicle was successfully tested in accordance with International standards as to its compliance with Electromagnetic Interference (EMI) Regulations. However, electromagnetic fields, such as those generated by radio and television transmitters, and cellular phones, can influence the functions of electric vehicles. Also, the electronics used in our vehicles can generate a low level of electromagnetic interference, which however will remain within the tolerance permitted by law. For these reasons we ask you to please observe the following precautions:



---

**WARNING: Danger of malfunction due to electromagnetic interference!**

- *Do not switch on or operate portable transceivers or communication devices (such as radio transceivers or cellular phones) when the vehicle is switched on!*
  - *Avoid getting near strong radio and television transmitters!*
  - *In case the vehicle should be set in motion unintentionally or the brakes are released, switch it off immediately!*
  - *Adding electrical accessories and other components or modifying the vehicle in any way can make it susceptible to electromagnetic interference. Keep in mind that there is no sure way to determine the effect such modifications will have on the overall immunity of the electronic system!*
  - *Report all occurrences of unintentional movement of the vehicle, or release of the electric brakes to the manufacturer!*
-

## 2.4 Safety Information on Driving and Freewheel Mode

---



### **Danger of injury if the vehicle tips over!**

- *Only ever negotiate gradients up to the maximum tilt-resistant gradient and only with the backrest in an upright position, and the seat lifter in the lowest position (if installed)!*
  - *Only ever drive downhill at a maximum of 2/3rds of the top speed! Avoid abrupt braking or accelerating on gradients!*
  - *If at all possible, avoid driving on slippery surfaces (such as snow, gravel, ice etc.) where there is a danger of you losing control over the vehicle, especially on a gradient! If driving on such a surface is inevitable, then always drive slowly and with the utmost caution!*
  - *Never attempt to overcome an obstacle when on an uphill or downhill gradient!*
  - *Never attempt to drive up or down a flight of steps!*
  - *Always approach obstacles straight on! Ensure that the front wheels and rear wheels move over the obstacle in one stroke, do not stop halfway! Do not exceed the maximum obstacle height (see Technical Specifications)!*
  - *Avoid shifting your centre of gravity as well as abrupt changes of direction when the vehicle is in motion!*
-



---

**Danger of injury if the vehicle tips over! (Continued)**

- *Never use the vehicle to transport more than one person!*
- *Do not exceed the maximum permissible load!*
- *When loading the vehicle, always distribute the weight evenly! Always try to keep the centre of gravity of the vehicle in the middle, and as close to the ground as possible!*
- *Note that the vehicle will brake or accelerate if you change the Driving Speed while it is in motion!*

**Danger of injury if you collide with an obstacle when driving through narrow passages such as doorways and entrances!**

- *Drive through narrow passages in the lowest Driving Speed and with due caution!*
-

### 3 Key features

- 1) Disengaging lever
- 2) Unlocking lever for sliding seat rails (front right below seat)
- 3) Unlocking lever for swivelling and removing seat (left below seat)
- 4) Operating console
- 5) Lever for adjusting steering column inclination
- 6) Keyswitch (ON/OFF)



## 4 Driving

### 4.1 Before driving for the first time...

Before you take your first trip, you should familiarise yourself well with the operation of the vehicle and with all operating elements. Take your time to test all functions and driving modes.



---

**NOTE:**

If installed, use the restraining systems (seat belts) each time you use the vehicle.

---

#### **Sitting Comfortably = Driving Safely**

**Before each trip, make sure that:**

- **You are within easy reach of all operating controls.**
- **The battery charge is sufficient for the distance intended to be covered.**
- **The restraining belt (if installed) is in perfect order.**

## 4.2 Taking Obstacles

Your Scooter can climb obstacles and kerbs of up to 5 cm in height.



---

### CAUTION: Danger of Tipping Over!

- *Never approach obstacles at an angle!*
  - *Put your backrest into an upright position before climbing an obstacle!*
- 


#### Driving up over an obstacle

- Approach the kerb or obstacle slowly head-on. Shortly before the front wheels touch the obstacle, increase the speed and reduce only after the rear wheels have also climbed the obstacle.

#### Driving down off of an obstacle

- Approach the kerb or obstacle slowly head-on. Before the front wheels touch the obstacle, reduce speed and keep it until also the rear wheels have come down off of the obstacle.

Correct

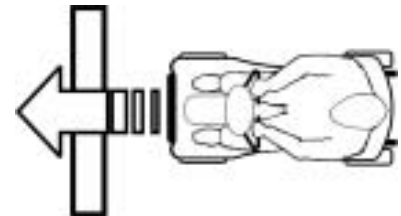

Incorrect



## 4.3 Driving up and down gradients

Your Scooter has a maximum hill-climbing ability without overturning of 6.8° (12 %)



---

**WARNING: Danger of tipping over!**

- *Only ever drive downhill at a maximum of 2/3rds of the top speed!*
  - *Always return the backrest of your seat to an upright position before ascending slopes! We recommend that you lean the backrest slightly to the rear before descending slopes!*
  - *Never attempt to ascend or descend a slope on slippery surfaces or where there is a danger of skidding (such as wet pavement, ice etc)!*
  - *Avoid trying to get out of the vehicle on an incline or a gradient!*
  - *Always drive in a straight direction along the road or path you are travelling on, rather than attempting to zigzag!*
  - *Never attempt to turn around on an incline or a slope!*
- 

## 4.4 Parking and stationary

**If you park your vehicle, or leave it at a standstill for a longer period:**

- switch off the power supply (keyswitch).

## 5 Pushing the scooter by hand

The motors of the scooter are equipped with automatic brakes, preventing the scooter from rolling away out of control when the power supply is switched off. When pushing the scooter, the magnetic brakes must be disengaged.

### 5.1 Disengaging Motors



---

**Danger of the vehicle running away!**

- *When the motors are disengaged (for push operation), the electromagnetic motor brakes are deactivated! When the vehicle is parked, the levers for engaging and disengaging the motors must without fail be locked firmly into the "DRIVE" position (electromagnetic motor brakes activated)!*
- 



---

**NOTE:**

The Scooter will start braking above a certain pushing or freewheeling speed if the drive has been disengaged. This is a safety function, and ensures that the Scooter cannot be pushed or freewheeled too quickly. In order to push the Scooter normally again, simply stop briefly and continue pushing at a lower speed.

---



**The lever for engaging and disengaging the motor is located on the right-hand side at the rear.**

**Disengaging the drive**

- Switch off the Scooter (keyswitch).
- Press the unlocking knob on the disengaging lever (1).
- Push the disengaging lever forwards. The drive is now disengaged.

**Engaging the drive**


- Pull the lever to the rear.



## 6 Operating console

### 6.1 Operating console arrangement

- 1) Battery charge display
- 2) Speed controller
- 3) Horn
- 4) Left-hand direction indicator (switches itself off automatically after 30 seconds)
- 5) Lighting
- 6) Status display
- 7) Warning blinker
- 8) Right-hand direction indicator (switches itself off automatically after 30 seconds)
- 9) Drive lever

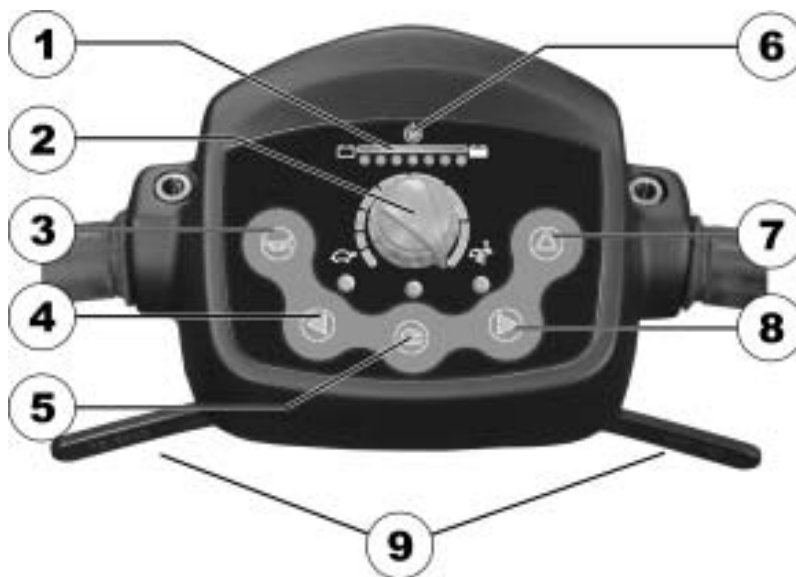

## 6.1.1 Status display



---

**NOTE:**

The ON/OFF diode is used as a fault display (status display). Chapter "**Error Codes and Diagnostic Codes**" on page **34** contains an explanation of the error codes.

---

## 6.1.2 Battery charge display

- All diodes illuminate: maximum driving range
  - Only red and yellow diodes illuminate: reduced driving range. Recharge the batteries at the end of your journey.
  - **Only red LEDs illuminate/blink, electronics beeps 3x: battery reserve = severely restricted driving range. Recharge batteries immediately!**
- 

**NOTE:**

Overdischarge protection: after a certain drive time on reserve battery power the electronics system switches the drive off automatically and brings the Scooter to a standstill. If you do not drive your Scooter for a while the batteries will "recuperate" and allow a further, but short, journey. However, after a very brief journey the red diodes will illuminate again and the electronic system will beep three times. This procedure leads to battery damage and should be avoided if possible!

---

## 6.2 Driving the Scooter

- Switch the power supply on (keyswitch)The operating console display illuminates. The Scooter is ready to drive.



---

**NOTE:**

If the Scooter is not ready to drive after switching on, check the status display (see Chapter "Status display" on page 27 and Chapter "Diagnosis and fault repair" on page 32).

---

- Set the required speed with the speed controller.
- Pull the right-hand drive lever carefully to travel forwards.
- Pull the left-hand drive lever carefully to travel in reverse.



---

**NOTE:**

The control system is programmed with standard values in the works. Your Invacare®dealer can carry out programming tailored to fit your requirements.

---



---

**Caution: any changes to the drive program can affect the driving characteristics and the tipping stability of the vehicle!**

- *Changes to the drive program may only be carried out by trained Invacare® specialist dealers!*
  - *Invacare® supplies all mobility products with a standard drive program ex-works. Invacare® can only give a warranty for safe vehicle driving behaviour - especially the tipping stability - for this standard drive program!*
-



---

**NOTE:**

To brake quickly, simply let go of the drive lever. It will then automatically return to the middle position. The Scooter will brake.

---


### 6.3 Activating and deactivating acoustic signals

The Scooter electronic system can be programmed to emit an acoustic signal in the following situations:

- battery capacity low (activated in delivery status)
- direction indicators actuated (deactivated in delivery status)
- hazard lamps actuated (deactivated in delivery status)

The electronic system must be switched off in order to activate or deactivate an acoustic signal for particular functions, and a particular keystroke combination needs to be entered when switching on again.

After a signal for a particular function has been successfully activated/deactivated, a combination of LEDs on the battery charge display will blink as a confirmation. The LEDs are numbered as shown on the right:

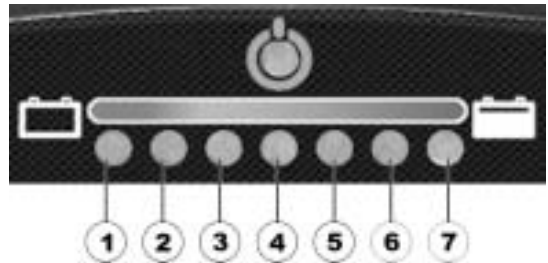

The keystroke combinations and LED codes for various options are as follows:

<b>Function:</b>	<b>Keystroke combination</b>	<b>LED(s)</b>	<b>Condition</b>
Acoustic signal at low battery capacity	"Lighting" + "direction indicator left"	D1	deactivated
		D1+D2	activated
Acoustic signal when direction indicators actuated	"Lighting" + "direction indicator right"	D3	deactivated
		D3+D4	activated
Acoustic signal when hazard lamps actuated	"Lighting" + "hazard lamps"	D5	deactivated
		D5+D6	activated

### **Activating or deactivating an acoustic signal**

Please proceed as follows to activate or deactivate an acoustic signal for a particular function:

- 1) Switch off the electronic system.
- 2) Enter the keystroke combination and hold.
- 3) Switch on the electronic system
- 4) Wait two seconds until the appropriate blink code is displayed on the battery charge display, then release the keys. Do not hold the keystroke combination down for more than five seconds.
- 5) If LED D7 blinks five times subsequently the acoustic signal has been successfully activated.
- 6) The electronic system will return to normal operating status automatically.

## 6.4 Diagnosis and fault repair

The electronic system offers diagnostic information to support the technician during the recognition and rectification of faults on the Scooter. If there is a fault, the status display blinks several times, pauses, then blinks again. The type of fault is displayed by the number of blinks in each group, which are also known as the "blink code".

The electronic system reacts differently depending on the seriousness of the fault and its effect on user safety. It can, for example:

- simply show the blink code as a warning and allow both driving and normal operation to continue;
- display the blink code, stop the Scooter and prevent further travel until the electronic system has been switched off and switched on again;
- display the blink code, stop the Scooter and not permit further travel until the fault has been rectified.

You can find detailed descriptions of individual blink codes, including possible causes and fault repair, in the section entitled "**Error Codes and Diagnostic Codes**" on page **34**.



## 6.4.1 Error diagnosis

If the Scooter shows a failure, please use the following guide to locate the fault.



---

**NOTE:**

Before making any diagnosis, ensure that the Scooter has been switched on at the keyswitch.

---

**If the status display is OFF:**

check whether the keyswitch is **SWITCHED ON**.

Check whether all cables are correctly connected.

**If the status bar indicator is BLINKING:**

count the number of blinks and then proceed to the next section.

## 6.5 Error Codes and Diagnostic Codes

Blink code	FAULT	Consequence for the Scooter	Comments
1	Battery must be charged	Continues to drive	<ul style="list-style-type: none"> <li>The batteries are discharged. Charge the battery as soon as possible.</li> </ul>
2	Battery voltage too low	Stops driving	<ul style="list-style-type: none"> <li>The batteries are depleted. Charge batteries.</li> <li>If you switch the Scooter off for a few minutes, the battery can often recuperate to such a stage that a short journey is still possible. You should only do this in emergency, however, because this causes the batteries to become excessively discharged!</li> </ul>
3	Battery voltage too high	Stops driving	<ul style="list-style-type: none"> <li>The battery voltage is too high. If the battery charger is connected, disconnect it from the Scooter.</li> <li>The electronic system charges the batteries when running downhill and when braking. This fault is caused when the battery voltage becomes too high during this process. Switch the Scooter off and on again. Switch on the lights to use up some power.</li> </ul>

<b>Blink code</b>	<b>FAULT</b>	<b>Consequence for the Scooter</b>	<b>Comments</b>
<b>4</b>	Power time exceeded	Stops driving	<ul style="list-style-type: none"> <li>The Scooter has used up too much power for a long period, probably because the motor has been overloaded or the Scooter has attempted to work against insurmountable resistance. Switch off the Scooter, wait a few minutes and then switch it on again.</li> <li>The electronic system has determined a motor short-circuit. Check the wiring harness for short-circuits and check the motor: Contact your Invacare® dealer.</li> </ul>
<b>5</b>	Brake failure	Stops driving	<ul style="list-style-type: none"> <li>Ensure that the disengaging lever is in the engaged position.</li> <li>There is a defect in the braking coil or in the cabling. Check the wheel lock and cabling for open or short-circuited circuits. Contact your Invacare® dealer.</li> </ul>
<b>6</b>	No neutral position when switching Scooter on.	Stops driving	<ul style="list-style-type: none"> <li>Drive lever is not in neutral when the keyswitch was turned. Put the drive lever in neutral, turn the power off and then turn on again.</li> <li>It may be necessary to recalibrate the drive lever. Contact your Invacare® dealer.</li> </ul>
<b>7</b>	Fault in speed potentiometer	Stops driving	<ul style="list-style-type: none"> <li>The drive lever electronics could be faulty or incorrectly connected. Contact your Invacare® dealer.</li> </ul>

<b>Blink code</b>	<b>FAULT</b>	<b>Consequence for the Scooter</b>	<b>Comments</b>
<b>8</b>	Motor voltage error	Stops driving	<ul style="list-style-type: none"> <li>The motor or its cabling is defective Contact your Invacare® dealer.</li> </ul>
<b>9</b>	Miscellaneous internal fault	Stops driving	<ul style="list-style-type: none"> <li>Contact your Invacare® dealer.</li> </ul>
<b>10</b>	Push mode / free running	Continues to drive	<ul style="list-style-type: none"> <li>The Scooter has exceeded the maximum speed for push mode / free running. Switch the electronics system off and on again.</li> </ul>

## 7 Adjustment facilities

### 7.1 Moving the seat position forwards or backwards

The disengaging lever for adjusting the seat is located front right below the seat

- Pull the lever (1) to disengage the seat.
- Slide the seat forwards or backwards into the required position.
- Let go the lever again to lock the seat into its required position.



## 7.2 Adjusting the armrest width

The hand wheels for releasing the armrests are located under the seat (1).

- Turn the hand wheels to loosen the fixing for the armrest.
- Adjust the armrests to the required width.
- Retighten the handwheels



### 7.3 Disengaging the seat to rotate it or remove it

The seat can be turned to one side to make getting in and out of the scooter easier. The seat is also easier to remove from this position.

The lever for disengaging the seat is located under the seat (1) on the right.

- Pull the lever forwards to disengage the seat.
- Turn the seat to one side.
- If you want to remove the seat, hold it firmly by the backrest and front edge and remove it upwards.



## 7.4 Adjusting the seat height

The seat height can be adjusted to 43, 45, 47, or 49 cm.



---

**CAUTION! Hazards due to unintentional disengaging of drive!**

- *Do not use the top drill hole! The seat is too low in this position and is not at a sufficient distance from the enclosure or disconnection lever!*
- 

**Requirements:**

- 2x open-ended spanners 17 mm
- 
- Remove the seat
  - Remove the battery and motor compartment cover.
  - Remove the seat pillar locking bolt using both open-ended spanners.





- Adjust the seat height.
- Reinsert the bolt and tighten.



## 8 Electrical System

### 8.1 Electronics Protection System

The vehicle's electronics are equipped with an overload-protection system.

If the motors are put under considerable strain for a longer period of time (for example, when driving up a steep hill) and especially when the ambient temperature is high, then the electronic system could overheat. In this case the vehicle's power is reduced gradually until it finally comes to a halt. The Status Display shows a corresponding error code (see chapter "**Error Codes and Diagnostic Codes**" on page 34). By switching the power supply off and back on again, the error code is cancelled and the electronics are switched back on. It will take approximately five minutes until the electronics have cooled down enough for the motors to restore full power again.

When the motors are stalled by an insurmountable obstacle, such as a high kerb, and the vehicle driver allows the motors to strain against this hindrance for more than 20 seconds without moving, then the electronics will automatically switch off to prevent the motors from being damaged. The Status Display shows a corresponding error code (see chapter "**Error Codes and Diagnostic Codes**" on page 34). By switching off and back on again, the error code is cancelled and the electronics are switched back on.

### 8.1.1 The main fuse

The entire electric system is protected against overload by two master fuses. The master fuses are mounted on the positive battery cables.



---

**NOTE**

A defective main fuse may be replaced only after checking the entire electric system. An Invacare® specialised dealer must perform the replacement. You can find information on the fuse type in chapter "**Technical Specifications**" starting on page **61**.

---

## 8.2 Batteries

### 8.2.1 What you need to know about batteries

Power is supplied by two 12V gel batteries. The batteries are maintenance-free and only need regular charging.

New batteries should always be fully charged once before their first use. New batteries will be at their full capacity after having run through approx. 10 - 20 charging cycles. How fast the batteries will be discharged will depend on many circumstances, such as ambient temperature, condition of the surface of the road, tyre pressure, weight of the driver, way of driving and utilisation of lighting, etc.



---

## NOTE

Gel batteries are not hazardous goods. This classification is based on the German *GGVS Hazardous Goods Road Transport Ordinances*, and the *IATA/DGR Hazardous Goods Rail Transport / Air Transport Ordinances*. Gel batteries may be transported without restrictions, whether by road, rail or by air. Individual transport companies have, however, guidelines which can possibly restrict or forbid certain transport procedures. Please ask the transport company regarding each individual case.

Pay attention to the Battery Charge Indicator! Make sure to charge the batteries when the Battery Charge Indicator shows that battery charge is low. We recommend charging the batteries after each trip, as well as each night over night. Depending on the level of discharge, it can take up to 12 hours until the batteries are fully charged again.

Protect your charger from sources of heat such as heaters and direct sunlight. If the battery charger overheats, charging current will be reduced and the charging process delayed.

To avoid damaging the batteries, never allow them to be fully discharged. Do not drive on heavily discharged batteries if it is not absolutely necessary, as this will strain the batteries unduly and shorten their life expectancy.

In case your vehicle is not used for a longer period of time, then the batteries must be charged at least once a month to maintain a full charge. Alternatively, the vehicle can stay connected to the charger. The batteries cannot be overcharged with the specified charger.

Please use only charging devices in Class 2. This class of chargers may be left unattended during charging. All charging devices which are supplied by Invacare® and comply with these requirements.

---

## 8.2.2 Charging the batteries

- Make sure you read and understand the battery charger's User's Manual, if supplied, as well as the safety notes on the front and rear panels of the charger!



---

**WARNING:****Danger of explosion and destruction of batteries if the wrong battery charger is used!**

- *Only ever use the battery charger supplied with your vehicle, or a charger that has been approved by Invacare®!*

**Danger of electric shock and damage to the battery charger if it is allowed to get wet!**

- *Protect the battery charger from water!*
- *Always charge in a dry environment!*

**Danger of short circuit and electric shock if the battery charger has been damaged!**

- *Do not use the battery charger if it has been dropped or damaged!*

**Danger of fire and electric shock if a damaged extension cable is used!**

- *Only ever use an extension cable if it is absolutely necessary! In case you must use one, make sure it is in good condition!*
-

**The charging socket is located on the left of the steering column**

**Connecting the battery charger**

- Switch off the Scooter.
- Push the protective cap on the side of the charging socket to the side.
- Connect the battery charger to the Scooter.
- Connect the battery charger to the mains.

**Disconnecting the battery charger from the Scooter**

- First unplug the battery charger from the mains.
- Then unplug the battery charger from the Scooter.



### 8.2.3 Removing and fitting batteries

---



**WARNING:**

**Danger of injury if the batteries are not handled correctly during assembly and maintenance work!**

- *New batteries should be installed by authorised technicians!*
- *Observe the warnings on the batteries!*
- *Take into account the heavy weight of the batteries!*
- *Only ever use the battery type defined in the technical specifications!*

**Danger of fire and burns if battery terminals are short-circuited!**

- *DO NOT short-circuit battery terminals with a tool!*
- 



**WARNING:**

**Corrosion and burns from acid leakage if batteries are damaged!**

- *Remove clothes that have been soiled by acid immediately!*

**After contact with skin:**

- *Immediately wash affected area with lots of water!*

**After contact with eyes:**

- *Immediately rinse eyes under running water for several minutes; consult a physician!*
-

### 8.2.3.1 Removing the batteries



---

**Requirements:**

- Open spanner, 11 mm.
- 
- Remove the seat.
  - Remove the battery and motor compartment cover.
  - Unplug the battery connecting plug (1).
  - Release the battery holder strap.
  - Remove the batteries.





- Loosen the blue cable on the negative battery terminal with the open-ended spanner, and remove the cable.
- Loosen the red cable battery clamp on the positive battery terminal with the open-ended spanner, and remove the cable.
- Repeat the procedure for the other battery



---

**NOTE:**  
Replacing new batteries takes place in reverse order.

---

### 8.2.3.2 How to handle damaged batteries correctly

---



**WARNING:**

**Corrosion and burns from acid leakage if batteries are damaged!**

- *Remove clothes that have been soiled by acid immediately!*

**After contact with skin:**

- *Immediately wash affected area with lots of water!*

**After contact with eyes:**

- *Immediately rinse eyes under running water for several minutes; consult a physician!*
- 



**Requirements:**

- Safety goggles
  - Acid-resistant gloves
  - Acid-resistant receptacle for transportation
- 

- Always wear appropriate safety clothing when handling damaged batteries.
- Place damaged batteries in an acid-resistant receptacle immediately after removing them.
- Only ever transport damaged batteries in an appropriate acid-resistant receptacle.
- Wash all objects that have come into contact with acid with lots of water.

**Disposing of dead or damaged batteries correctly**

Dead or damaged batteries can be given back to your dealer or directly to Invacare®.

## 9 Care and maintenance

---

**NOTE:**

Have your vehicle checked once a year by an authorised Invacare® dealer in order to maintain it's driving safety and roadworthiness.

---

**Cleaning the vehicle**

When cleaning the vehicle, pay attention to the following points:

- Only use a damp cloth and gentle detergent.
- Do not use any abrasive or scouring liquids.
- Do not subject the electronic components to any direct contact with water.
- Do not use high-pressure cleaning devices.

**Disinfection**

Spray or wipe disinfection using a tested and recognised product is permitted. A list of the current permitted disinfectants is available from the Robert Koch Institute at <http://www.rki.de>.

<b>Maintenance Jobs</b>	When Delivered	Weekly	Monthly
<b>Seat and backrest padding:</b>			
• Check for perfect condition.			✓
<b>Tyres:</b>			
• Have tyres checked for specified air pressure (2,5 bar).	✓	✓	
<b>Front wheels</b>			
• Front wheels must spin smoothly.		✓	
• If wheels wobble or do not spin easily, adjust steering pivot pin or front wheel bearing.			✓
<b>Rear wheels:</b>			
• Test wheel for firm seat on the axle drive shaft.			✓
• Rear wheels must spin without wobbling			✓
<b>Electronics / Electrical System:</b>			
• Check all plug connections for condition and firm connection.			✓
• Have batteries been fully charged before the daily operation?	Before every trip		
• Are all holders, screws firmly fixed, tight and safe?			✓
• Are all electric bulbs of the lighting system (if applicable) in working order?	Before each trip		
<b>Cleaning:</b>			
• Clean all parts carefully.	When necessary		

**Once a year you should have your vehicle inspected and serviced by your authorised dealer. A complete checklist of necessary maintenance work can be found in the Service Manual, which can be obtained from Invacare®.**

## 10 Repair Instructions

The following are instructions on repairs that can be performed by the user. For the specifications of spare parts please see "**Technical Specifications**" on page **61**, or consult the Service Manual, available from Invacare® (in this connection please see the addresses and phone numbers in section "**How can you get in touch with Invacare®?**" on page **2**). In case you require assistance, please contact your Invacare® Dealer.

### 10.1 Repairing a flat tyre



---

**WARNING: Danger of damage or injury if the vehicle is accidentally set into motion during repairs!**

- *Switch the power off (ON/OFF Button)!*
  - *Engage the motors!*
  - *Secure the vehicle against rolling away by placing wedges under the wheels!*
-

### 10.1.1 Mending punctures (pneumatic tyres of type 3.00 - 4)

---



#### Requirements:

- Open spanner, 17 mm.
  - Rubber hammer
- 

- Raise the vehicle and place wooden blocks underneath to support it.
- Remove the wheel locknut (1) with a 17 mm open-ended spanner.
- Remove the wheel by tapping it gently with the rubber hammer on the rear face to carefully loosen it from the axle.



#### Problems when removing wheel?

It may be necessary to use a special tool Please ask your Invacare® dealer to help you.

---

### 10.1.1.1 Repairing punctured tyres



---

#### Requirements

- inner tube repair set **or** a new inner tube
  - talcum powder
  - open-ended spanner, 13 mm
  - jaw spanner 13 mm
- 

- Remove valve cap.
- De-inflate the tyre by pressing in the centre valve pin.
- Loosen the 4 nuts (1) with the socket spanner and remove them. Counter the bolts from the other side with the jaw spanner to prevent turning.
- Remove both wheel rim halves out of the tyre and remove the inner tube.
- Repair the inner tube and refit in the wheel, or replace it with a new inner tube.





---

**Did the old inner tube get wet during the repair?**

If you repaired the old inner tube and reused it, and it became wet during repair, it is much easier to refit it into the wheel if you powder it lightly with talcum powder.

---

- Refit the wheel rim parts from outside into the tyre.
- Pump up the tyre lightly.
- Reinsert the nuts and bolts which hold the wheel rim together and tighten fully.
- Make sure that the tyre is properly located on the wheel rim.
- Inflate the tyre up to the recommended tyre pressure (2.5 bar or 40 psi).
- Check to make sure that the tyre is still located properly on the wheel rim.
- Screw the valve cap back on.
- Reassemble the wheel.



## 11 Dismantling the Scooter for transport

Please proceed as follows to dismantle the Scooter for transport:

- Remove the seat
- Take out batteries
- Remove the drive unit

Please proceed in reverse order to reassemble the Scooter.

### 11.1 Removing the seat

The lever for releasing the seat so that the seat can be rotated and removed is located on the left under the seat (1).

- Press the lever forwards to disengage the seat.
- Turn the seat to the side.
- To remove the seat, hold it firmly by the backrest and the front edge and pull it upwards.



## 11.2 Removing the battery and drive unit

- Once you have removed the seat, remove the battery and motor compartment cover panel. The cover panel is secured using a velcro band, so you only need to pull it upwards to remove it.

### 11.2.1 Removing the batteries


Please see chapter "**Removing and fitting batteries**" on page 47 for information on removing the batteries.

### 11.2.2 Removing the drive unit

- Separate the main wiring harness connecting plug (1).



- Pull the disengaging lever (A) to the rear. The drive unit lock is now released.
- Lift up the chassis with the seating tube. The drive unit can now be released from the chassis.

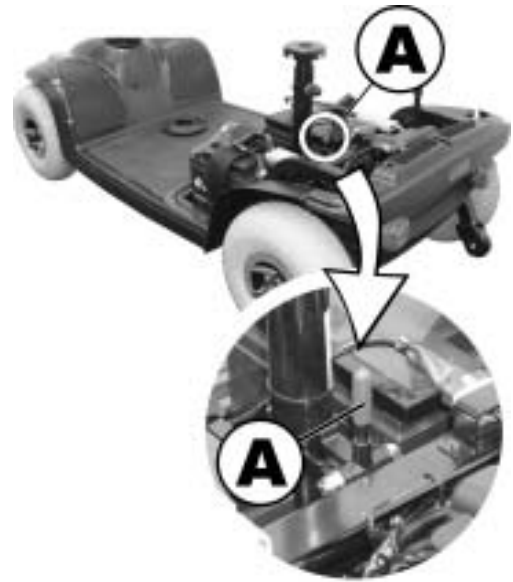

## 12 Disposal

- The equipment wrapping is potentially recyclable.
- The metal parts are used for scrap metal recycling.
- The plastic parts are used for plastic recycling.
- Electric components and printed circuit boards are disposed of as electronic scrap.
- Disposal must be carried out in accordance with the respective national legal provisions.
- Ask your city or district council for details of the local waste management companies.

## 13 Technical Specifications

<b>Permissible Operating and Storage Conditions</b>	
Temperature range for operation according to ISO 7176-9:	<ul style="list-style-type: none"><li>• -25 ... +50 °C</li></ul>
Temperature range for storage according to ISO 7176-9:	<ul style="list-style-type: none"><li>• -40 ... +65 °C</li></ul>

<b>Electrical system</b>	
Motors	<ul style="list-style-type: none"><li>• 1 x 240 W</li></ul>
Batteries	<ul style="list-style-type: none"><li>• 2 x 12V / 36 Ah, leakproof/gel</li></ul>
Main fuse	<ul style="list-style-type: none"><li>• 70 A</li></ul>

<b>Charging device</b>	
Output current	<ul style="list-style-type: none"><li>• 5A ±</li></ul>
Output voltage	<ul style="list-style-type: none"><li>• 24V nominal (12 cells)</li></ul>
Input voltage	<ul style="list-style-type: none"><li>• 200 – 250V nominal</li></ul>
Operating temperature (surroundings)	<ul style="list-style-type: none"><li>• -25° ... +50°C</li></ul>
Storage temperature	<ul style="list-style-type: none"><li>• -40° ... +65°C</li></ul>

<b>Tyres</b>	
Tyre type	<ul style="list-style-type: none"><li>• 3.00 - 4 pneumatic</li></ul>
Tyre pressure	<ul style="list-style-type: none"><li>• 2.8 bar</li></ul>

<b>Driving characteristics</b>	
Speed	<ul style="list-style-type: none"><li>• 8 km/h</li></ul>

<b>Driving characteristics</b>	
Max. hill-climbing ability without overturning	• 17.5% (10°)
Max. climbable obstacle height	• 6 cm
Turning radius	• 65.5 cm
Drive range in accordance with ISO 7176 ***	• 36 km

<b>Dimensions</b>	
Overall length	• 122 cm
Drive unit width	• 59 cm
Overall width (armrest adjustment range)	• 58 ... 73 cm
Total height	• 99 cm
Seat width	• 47 cm
Seat depth	• 41 cm
Backrest height ****	• 47.5 cm
Backrest angle	• +9.5°
Armrest height	• 20 cm

<b>Weight</b>	
Empty weight	• 83.5 kg

<b>Loading</b>	
Max. load	• 136 kg

<b>Axle loads</b>	
Max. front axle load	• 85 kg
Max. rear axle load	• 160 kg

\*\*\* Note: The drive range of an electric wheelchair is strongly influenced by external factors, such as the charging state of the batteries, surrounding temperature, local topography, road surface characteristics, tyre pressure, weight of driver, drive style and use of batteries for lighting, servos etc.

\*\*\*\* Measured without seat cushion

## 14 Inspections Performed

It is confirmed by stamp and signature that all jobs listed in the inspection schedule of the Service and Repair Instructions have been properly performed. The list of the inspection jobs to be performed can be found in the Service Manual which is available through Invacare®.

<u>Delivery Inspection</u>	<u>1<sup>st</sup> Annual Inspection</u>
Stamp of authorised Dealer / Date / Signature	Stamp of authorised Dealer / Date / Signature
<u>2<sup>nd</sup> Annual Inspection</u>	<u>3<sup>rd</sup> Annual Inspection</u>
Stamp of authorised Dealer / Date / Signature	Stamp of authorised Dealer / Date / Signature
<u>4<sup>th</sup> Annual Inspection</u>	<u>5<sup>th</sup> Annual Inspection</u>
Stamp of authorised Dealer / Date / Signature	Stamp of authorised Dealer / Date / Signature



## **Hersteller / Manufacturer/fabricante:**

### **CHIEN TI ENTERPRISE CO. LTD.**

No. 13, Lane 227, Fu Ying Road  
Hsin Chuang, Taipei, Taiwan  
**R.O.C.**

## **Inverkehrbringer / European Representative / representante europeo:**

### **K&K advice**

Loswal 2  
8433 HD Haulerwijk  
**Netherlands**

## **Distribution/Distribución:**

### **France, Italia, España, Portugal**

#### **Invacare® Poirier SAS**

Route de St Roch  
F-37230 Fondettes

#### **France**

Tel: (33) (0) 247 - 62 64 66, Fax: (33) (0) 247 - 42 12 24

### **United Kingdom, Ireland**

#### **Invacare® Ltd**

South Road  
Bridgend Industrial Estate  
Mid Glamorgan - CF31-3PY

#### **United Kingdom**

Tel: (44) (0) 1656 - 664 321, Fax: (44) (0) 1656 - 667532

**Sweden, Danmark, Nederland**

**Dolomite AB**

Box 55

V. Götgatan 5

S-33421 Anderstorp

**Sweden**

Tel: (46) (0) 371 - 58 84 00, Fax: (46) (0) 371 - 170 90

**Deutschland, Eastern Europe**

**Invacare Deutschland GmbH**

Kleiststraße 49

32457 Porta Westfalica

**Deutschland**

Tel: (49) (0) 5731 - 754 0, Fax: (49) (0) 5731 - 754 52 191

Order No. of this Manual: 1506942.DOC  
Release Date: 29.03.07

**English**