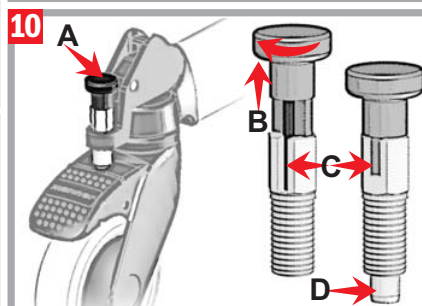
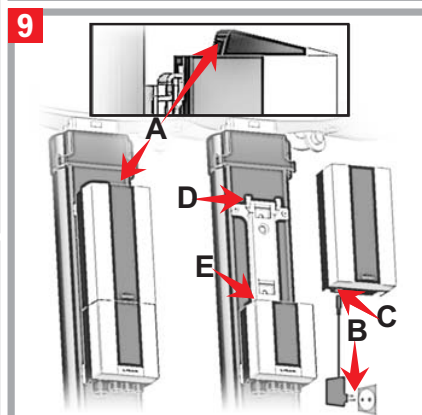
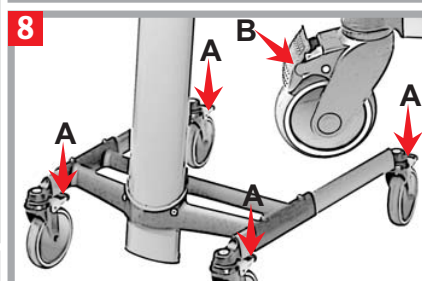
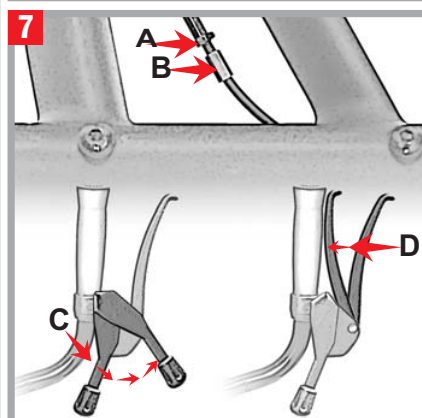
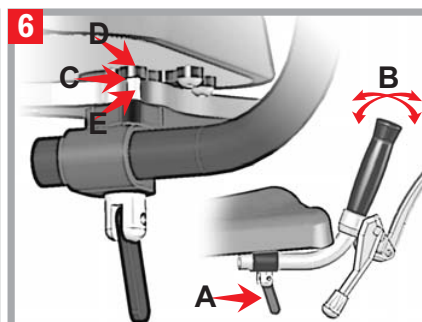
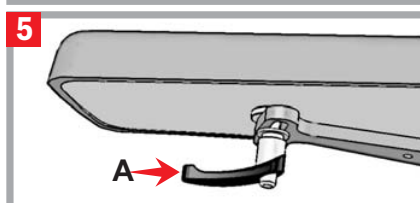
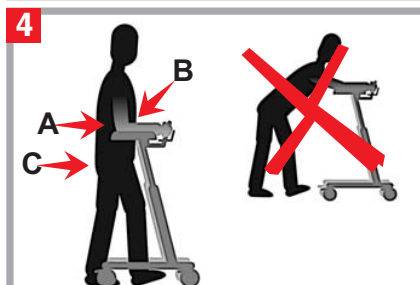
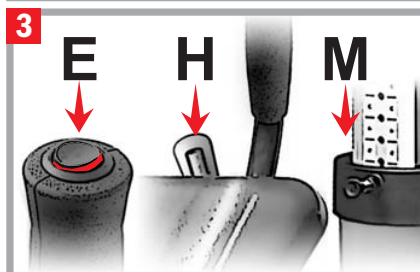
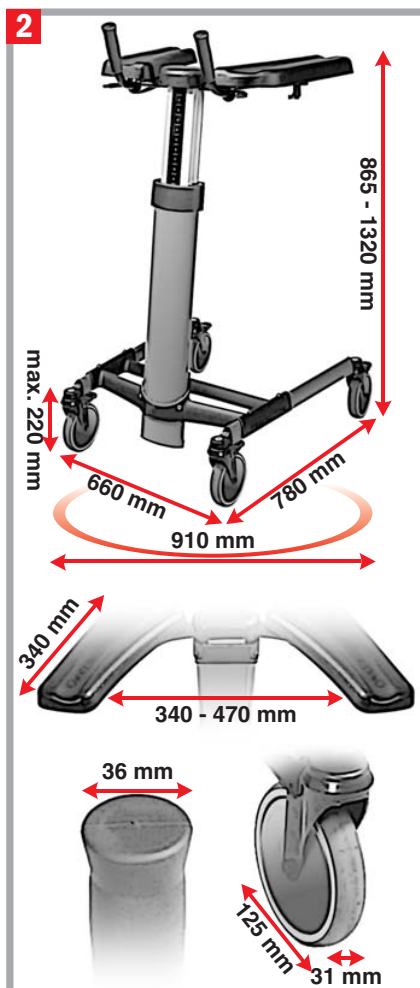
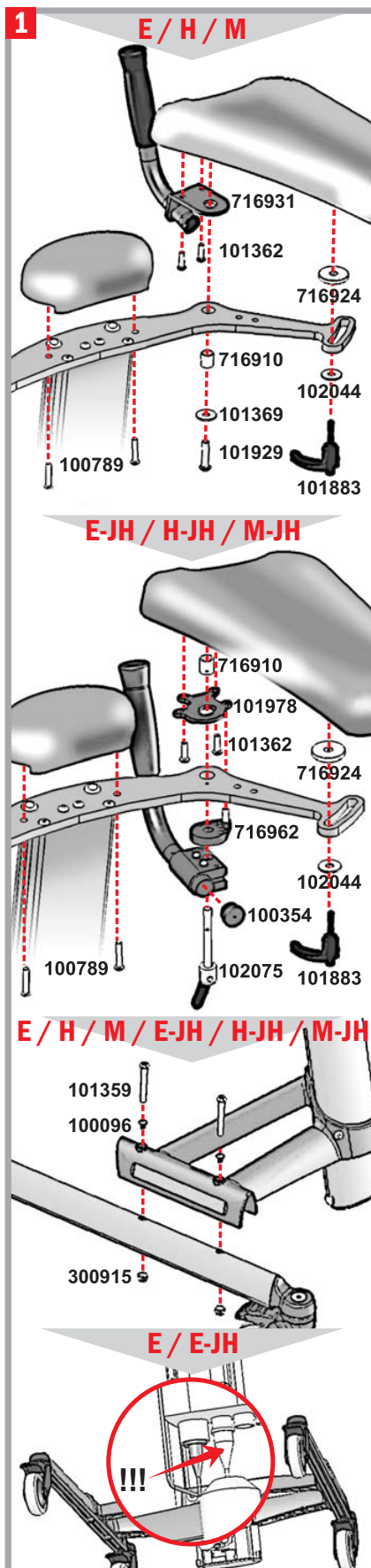
Bruksanvisning 2 - 4 NO

User manual 5 - 7 EN

Bedienungsanleitung 8-10 DE

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Welcome as a user of a TOPRO PRODUCT

Every TOPRO product is developed, designed and made in Norway. Advanced manufacturing equipment, quality and environmental security gives great accuracy and maximum quality in all levels of production.

Guarantee

TOPRO TAURUS is guaranteed free for faults and defects for 2 years. An exception is if the damage is caused by incorrect use or parts that are exposed to natural wear and tear (such as e.g. brakes, wheels and handles). For repairs in the guarantee period, please refer to your local mobility shop or dealer. The product is declared to be in conformity with the Medical Device Directive 93/42/EEC.

The texts next to the red squares are explanations to the illustrations on the inside of the cover.

1 Assembling and adjusting

Some of the models need assembling. TOPRO TOPRO TAURUS H-JHB, E-JHB and M-JHB are delivered completely assembled. For descriptions of the models see figure 3.

Adjust the forearm supports and handles, see figure 5. The battery must be charged for TOPRO TAURUS E, see figure 9.

2 Measurements

3 Standard models

TOPRO TAURUS E. Weight 24 kg. Electric height adjustment.

Press the button B on top of the handle to the left or to the right to regulate the height.

TOPRO TAURUS H. Weight 18 kg. Hydraulic height adjustment.

Pull the handle up. Use a little body weight to press down or reduce the counter weight to increase the height. Release the handle at the required height.

TOPRO TAURUS M. Weight 16 kg. Manual height adjustment.

Pull out the index pin and adjust to required height. Push the pin up and down until it locks in a suitable hole on the scale. This can be heavy to do on your own as the top of the walker must be held up.

All models are delivered as standard editions, plus two variants with the following supplementary equipment:

JH Adjustable handles.

JHB Adjustable handles. Driving and parking brakes. Swing lock (directional block).

	TOPRO ART. No.
TOPRO TAURUS E	814790
TOPRO TAURUS E-JH	814788
TOPRO TAURUS E-JHB	814789
TOPRO TAURUS H	814785
TOPRO TAURUS H-JH	814786
TOPRO TAURUS H-JHB	814787
TOPRO TAURUS M	814780
TOPRO TAURUS M-JH	814781
TOPRO TAURUS M-JHB	814782

4 Important to know about safety and use

Notify your mobility shop or dealer immediately if the product is damaged when received.

Max user weight is 150 kg. Max user height 2 m. Do not let children play with the walker.

The walker must be adjusted to the recommended height for right support and safe walking: Stand with relaxed shoulders and have your elbow at a 90 degree angle **A**. Hold the walker as close to your body to maintain the correct support and a safe walk **B**. The easiest way to keep your balance is to walk straight, look forward and hold your backside in **C**. Do not push the walker too far from your body as this can cause unnecessary strain and you could fall. Do not push the walker forward if the front wheels have stopped at an obstacle. The walker may tilt and cause an accident.

TOPRO TAURUS can be used inside and outdoors, but only on a levelled surface.

NB: TOPRO TAURUS E must not be used in a damp environment.

5 Adjusting the forearm support

The forearm supports are adjusted individually. Unscrew the lock underneath the forearm support **A** until the cushion is moveable. Place at the required position and tighten the screw. **NB** If the lock sticks out, it can be dangerous. Pull the lever down and rotate it so that it doesn't stick out.

6 Angle adjustment and twisting the handle

(TOPRO TAURUS E-JH / H-JH / M-JH / E-JHB / H-JHB / M-JHB)

Unscrew the screw **A** until you can move the handle. Adjust the handle **B** to the required angle, length and twist and tighten the screw. **NB** The screw must not be screwed too far out as the parts will fall from each other. If this happens, then put the pin **C** in the hole **D** and **E**.

See figure 1 for more detailed instructions.

7 Brakes

(TOPRO TAURUS E-JHB / H-JHB / M-JHB)

Adjusting the brakes.

Unscrew the nut **A**. Tighten / loosen the brake wire with the adjusting screw **B**. Make sure the brakes work properly and don't sweep when brakes are off. Adjust both sides.

Using the parking brakes.

The brake lever **C** works on both rear wheels simultaneously. Push the brake lever forward. It has some resistance before it stops and locks both the rear wheels. Pull the brake lever backwards to release the rear wheels. Inspect the brakes frequently to make sure they work properly.

Using the driving brakes.

The brake lever **D** works on both rear wheels simultaneously. Pull the brake lever towards you to stop the rear wheels. When the brake lever is released, the rear wheels are released. Inspect the brakes frequently by following the information above.

8 Using the foot brakes

Step on the foot brakes **A** on the rear wheels or when required on all wheels until you hear a clicking sound. The wheels are now in a locked position **B**. To release the wheels, step on the back of the foot brake pedal so that it comes up again.

9 Charging the battery

(TOPRO TAURUS E / E-JH / E-JHB)

The battery can not be charged incorrectly. It can be charged at any time without doing any harm. We recommend the battery to be charged every evening. You will hear a short sound when 50% of the battery's capacity is used. It will take approximately 6-7 hours to fully charge. To release the battery, press the release button on top **A** and pull up. Plug your charger into the wall socket **B**, and then plug the other end into the battery's charging point **C**. To attach the battery again, place it on the hooks on the mounting bracket **D** while you steer it down the slot **E**. Press the battery towards the pile until you hear a clicking sound. The battery must sit tight and not be movable.

10 Locking the swing (directional block)

(TOPRO TAURUS E-JHB / H-JHB / M-JHB)

To be done with an index pin **A** on both wheels. Make sure the mounting bracket for the foot brake is pointing backwards.

This is how the index pin works.

Locking: Twist the upper part of the index pin **B** so that it fits into the track of the lower part **C**. By doing this the pin **D** will slide out and lock.

Releasing: Pull the upper part of the index pin **B** up and turn it 90 degrees. It will then lock in the upper position. The pin **D** slides in and the index pin is unlocked.

Material

The walker is made of plastic coated aluminium pipe. The forearm cushion is of polyurethane.

Cleaning

The walker can be cleaned with household detergents. TOPRO TAURUS H and M can be hosed. TOPRO TAURUS E can not be hosed because of the electric motor.

Cleaning with disinfections must be left to experienced personnel

To disinfect the walker, there must be used disinfection containing 70 - 80 % Ethanol. We advise against the use of disinfections containing chloride and phenol. The manufacturer cannot be incriminated if damages occur under disinfecting caused by wrong use or when cleaned by unauthorized personnel.

Inspection / maintenance

The wheel bearings are sealed and maintenance-free. All the other parts do not need to be lubricated. It is recommended that general inspection and maintenance is carried out and that the items below are checked frequently: tubes and frame, handles and forearm supports, brakes, wheels and accessories.

When something does not work

1 = possible reason.

2 = action.

Brakes

1 Worn out / defect wheel or brake.

2 Change the complete wheel.

1 Unadjusted brake.

2 Adjust the brake wire.

1 Worn out / defect brake mechanism on wheel.

2 Change the brake mechanism.

1 Worn out / defect brake handle.

2 Change the complete brake handle.

1 Worn out / damaged brake wire.

2 Change the brake wire.

Height adjustment TOPRO TAURUS H

1 The gas spring is worn out or damaged.

2 Change the gas spring.

1 The control arm is maladjusted.

2 Adjust the control arm.

Height adjustment TOPRO TAURUS E

1 Battery is flat.

2 Charge or change the battery.

1 The battery is not correctly assembled.

2 Check the battery bracket.

1 Bad contact between the motor and battery.

2 Check the contacts and the cables.

1 Defect switch.

2 Change the switch.

1 Defect height adjusting panel.

2 Change the height adjusting panel.

Accessories

For pictures see the inside of the cover.

	TOPRO ART.No.
A Height adjustment panel (only TOPRO TAURUS E)	814805
B Stand for body fluid-boxes	814034
C Small basket	814821
D Table	814810
E Footplate	814645
F Holder for oxygen	814820
G Infusion stand	814793
H Adjustable handles	814001

The accessories are to be ordered separately to give the possibility to compose a TOPRO TAURUS that suits the individual users need. Contact your mobility shop, dealer or TOPRO for an updated overview of the accessories, or visit our homepage www.topro.no

In case of faults or questions please contact your mobility shop or dealer.

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