



Steerable Knee Walker



VP186

Fixing and Maintenance Instructions

INTRODUCTION

Thank you for purchasing the Steerable Knee Walker from Aidapt.

These instructions should be read carefully and left with the user of the product for future reference.

Please ensure this equipment is installed by a competent person.

VP186 – Steerable Knee Walker

Do not exceed the weight limit stated as 250 lbs (17.8 St) – doing so could put you at risk.

Do not use or combine bags or baskets from other manufacturers.

INTENDED USE

The Steerable Knee Walker is an innovative tool to aid recovery from foot injury, breaks, sprains and other mobile related ailments. The Knee Walker allows the user to evenly distribute their body weight on the device. It is supplied with a cushioned platform to support the injured leg and ankle to ensure no weight is placed on the lower leg.

The Knee Walker is designed for easy steering and stopping. It features brakes for controlled movement while operating the device.

- Please note placement of the leg on the pad should be centred and positioned forward to ensure the leg covers the full length of the pad.
- Ensure the user stands as straight as possible while using the device.
- Ensure the injured foot is pointing downwards.
- Start with small steps, with practice the user should be able to move forward without deviating the opposite side.

FEATURES

- Compact for easy transport.
- Height adjustable.
- Benefits users recovering from foot surgery, breaks, sprains, amputation, and ulcers of the foot.
- Sturdy dual frame adds stability and durability.
- Can be steered for increased manoeuvrability.

ASSEMBLY INSTRUCTIONS

Please examine your product for any visible damage before using/installing. If you see any damage or suspect a fault, please do not use, but contact your supplier.

1. Remove all contents from the carton.
2. Straighten the tiller to the upright position. Slide the locking mechanism to the side to allow tiller to reach its full upright position (*fig. 1*). Release locking mechanism to lock tiller in place (*fig. 2*).
3. Slide the locking mechanism into the tiller groove, depress the locking handle to secure and lock (*fig. 3*).
4. Insert pad post into the frame; secure the pad in place by inserting locking pin (*figs. 4 & 5*). Lock pin in place by sliding the tab into position (*figs. 6 & 7*).
5. Lock pad in place by using locking lever, turning it clockwise to tighten, before folding into the frame to secure (*figs. 8 & 9*).

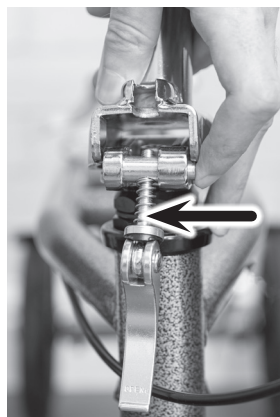


fig. 1



fig. 2

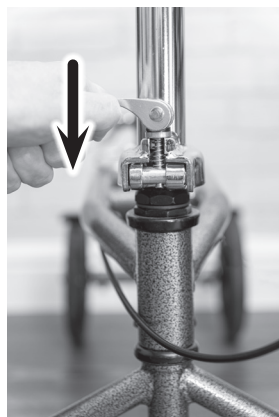


fig. 3

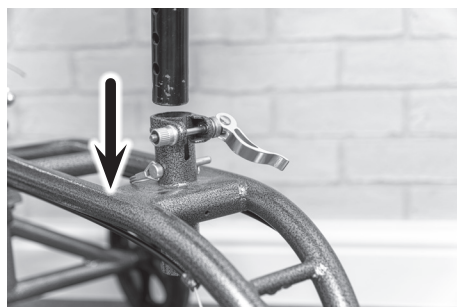


fig. 4



fig. 5

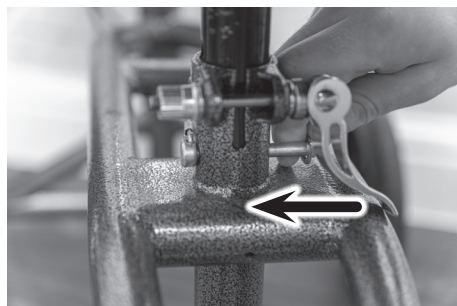


fig. 6



fig. 7



fig. 8



fig. 9

BASKET

- Your Knee Walker is supplied with a basket; attach by sliding basket receptacles down over hooks on the front of the tiller (*figs. 10 & 11*).



fig. 10



fig. 11

OPERATING THE BRAKES

Ensure the Knee Walker brake system is working properly before use.

- The brake system works similar to a bicycle and also has a locking feature
- To use, simply pull the brake lever with your finger tips and pull towards the handlebar (*fig. 12*).
- In order to lock the brake, pull lever towards the handlebar and push down on the spring-loaded push button (*fig. 13*).
- To release brake, pull lever towards the handlebar (*fig. 14*). The pushpin should automatically pop up (*fig. 15*).



fig. 12



fig. 13



fig. 14



fig. 15

ADJUSTING THE HAND BRAKE

- For minor adjustment, the brake adjuster located on the handbrake can be turned out from the brake counter clockwise to tighten the brake, or clockwise to loosen the brake. Please ensure you keep the cable adjuster nut against the hand brake (*figs. 16 & 17*).
- If further adjustment is required, apply the same method of adjustment at the lower cable adjuster (*figs. 18 & 19*).
- As you adjust the cable and the adjuster nut away from each other the brakes will tighten, and as you adjust them closer the brakes will loosen.

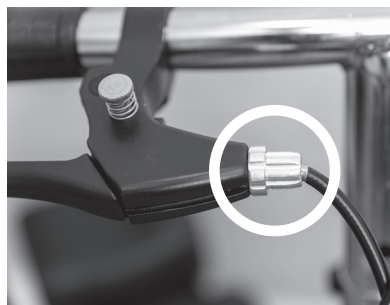


fig. 16

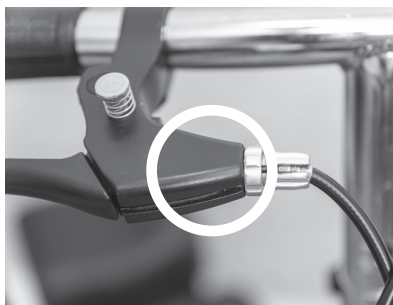


fig. 17



fig. 18

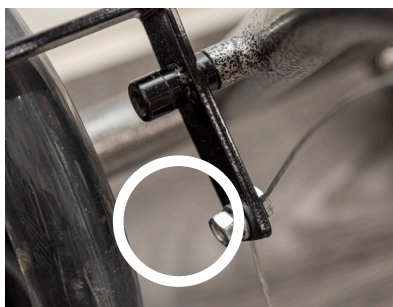


fig. 19

ADJUSTING THE HANDLE HEIGHT

- To adjust, loosen the adjustment screw on the handle by turning it counter clockwise until it can be removed (**fig. 20**).
- Adjust height to desired height (**fig. 21**), reinstall the adjustment screw by turning it clockwise until it is tight (**fig. 22**).



fig. 20



fig. 21



fig. 22

ADJUSTING THE PAD HEIGHT

- To adjust, loosen locking lever located under the pad, remove the locking pin.
- Adjust pad to desired height and then reinstall locking pin, secure in place with retainer tab.
- Tighten the locking lever to lock pad in place.

FOLDING TILLER

To fold the tiller, release the tiller-locking handle and slide assembly to the side until tiller is disengaged. Fold tiller down until it rests on the pad and release locking mechanism.

CAUTIONS

- Please contact your provider if any parts are loose or show excessive wear.
- Do not operate on loose gravel or uneven surfaces.
- Do not travel over large cracks or fissures.
- Move forward slowly when encountering cracks or separations in walkway surfaces.
- Do not operate in excess of 3 miles per hour.
- Do not turn unless one foot is on the ground.
- Do not turn at high speeds.
- Do not operate on incline about 15 degrees.
- Do not travel over drop or curbs.
- This product is not a toy.
- Always check fasteners on the tiller, wheels, brakes and pad for tightness before use.
- Adjust steering mechanism to the correct height.
- Do not sit, stand or lean on steering mechanism.
- Steering mechanism is not weight bearing.

CLEANING

All of the product components are manufactured from quality materials. However, it is very important that you only clean your Knee Walker using a non-abrasive cleaner or mild detergent with a soft cloth avoid using lime scale remover. Abrasive cleaners or abrasive cleaning pads will seriously damage your product beyond repair.

If to be disinfected by heat, one of the following three temperatures and duration of exposition can be used:

- a. a temperature of 90°C for 1 minute.
- b. a temperature of 85°C for 3 minutes.
- c. a temperature of 80°C for 10 minutes.

CARE, MAINTENANCE AND YOUR OBLIGATION

1. Only use your Aidapt Knee Walker for the purpose for which is intended, which is as an aid to walking. It is intended your Knee Walker is only used on standard pavements or at home. It should not be used on rough terrain or pot-holed surfaces. The use of your Knee Walker on these surfaces can seriously compromise your safety and may damage your Knee Walker beyond repair.
2. Keep your Aidapt Knee Walker clean by wiping down with a damp cloth with a mild detergent. Do not under any circumstances use an abrasive or solvent-based cleaner, as this will damage the Knee Walker and void your warranty.
3. Wipe off all surfaces with a dry cloth and use a water dispersing agent such as WD40 on all metal joints.
4. Check that all wheels function correctly and that all wheels move freely and without obstruction.

Withdraw your Aidapt Knee Walker immediately if any of the wheels become difficult to turn or if you suspect damage or any other fault.

5. Monthly checks – check that the side locking levers are poorly secure (*figs. 3 & 9*). Check the adjustment to your brakes (*figs. 16 - 19*). Ensure that the brakes are correctly adjusted.
6. Do not fit any parts or accessories to your Aidapt Knee Walker manufactured by another company. To do so will annul your warranty and may seriously compromise your safety.
7. We suggest that you check your Aidapt Knee Walker visually every day.

Please check the Knee Walker periodically for any signs of damage and if in doubt contact your supplier.

TECHNICAL INFORMATION

Width: 410mm

Depth: 762mm

Height: 806mm

Net Weight: 9.8kg

Gross Weight: 12.25kg

SERVICE WARRANTY

Aidapt Bathrooms Ltd guarantees the product free from defects in material and workmanship for a period of one year.

Should this product be operated under conditions other than those recommended, or any attempts made to service or modify the product, then the warranty is rendered void.

The product you buy may differ slightly from illustrations. This warranty is in addition to, and does not affect your statutory rights.

Our guarantee is administered by our retailers.

If your product arrives damaged, you must contact the retailer from whom you bought it. The retailers contact details will be on the invoice that arrived with the product, or on the email you received when you placed the order. Do not contact Aidapt Bathrooms Ltd, only your retailer can arrange a replacement or refund.

If your product fails within the guarantee period, please contact the retailer from whom you bought it.

If you have received your product and require technical help, please call our help desk on 01744 745 000.

A copy of the instructions leaflet can be downloaded from our website.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX

Telephone: +44 (0) 1744 745 020 Fax: +44 (0) 1744 745 001

Web: www.aidapt.com

Email: sales@aidapt.co.uk