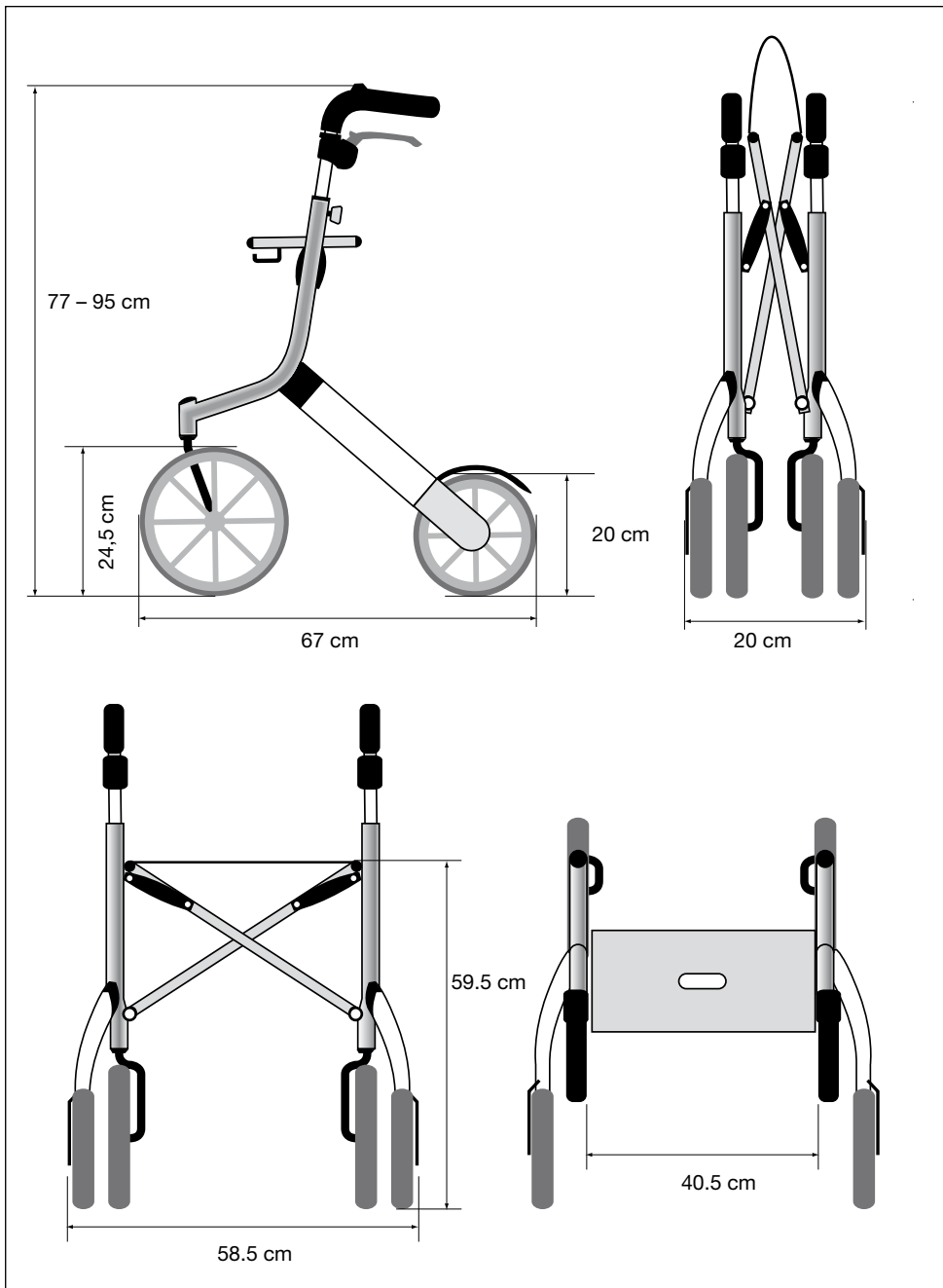
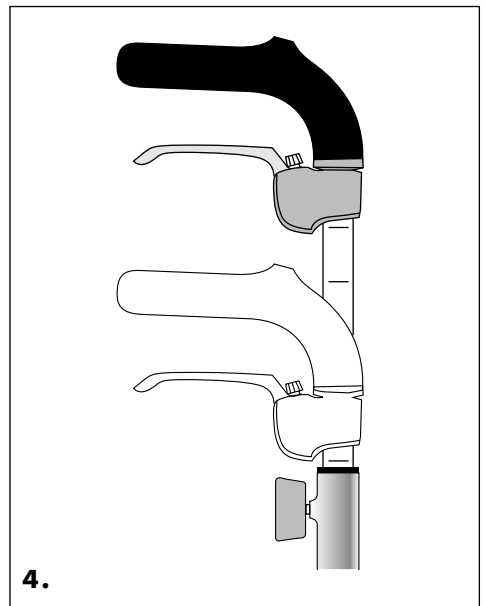
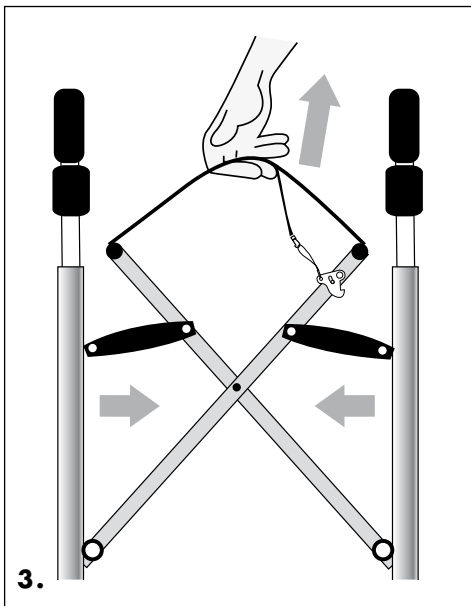
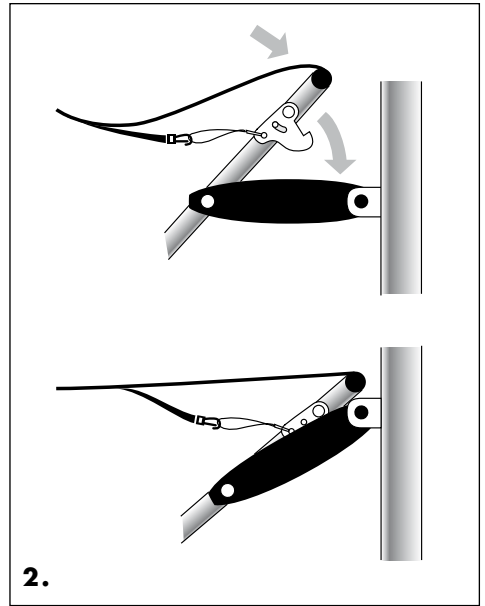
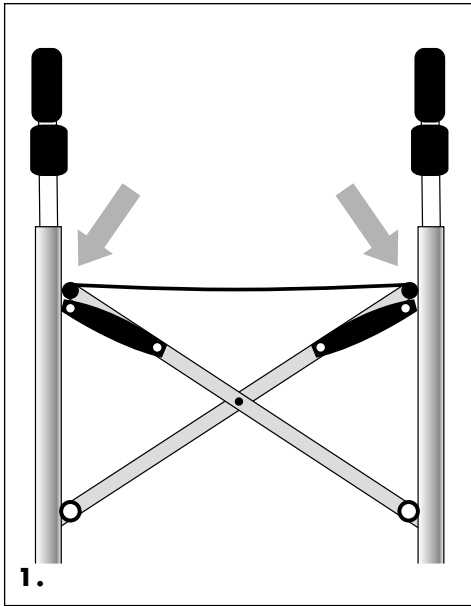
LET'S GO OUT

Lets Go udendørs rollator

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Belasting tas (optioneel) max 12 kg.

Let's Go Out Outdoor Walker.

Congratulations to you for purchasing your new outdoor rollator that will make your life a little easier, more pleasant and more safe.

To make it easier for you to get familiar with your rollator it is important that you study this manual.

The rollator helps you to walk and move outdoors. You can also use it to sit on when you want to rest during a walk. Then do not forget to activate the parking brake – position 3 on the picture to the right.

The large front wheels make it easier for you to cross kerbs or other obstacles. Make sure to cross kerbs in a right angle so the rollator will not be unsteady during crossing.

The seat is designed only to sit on, not for transport of for instance children or objects. Do not put heavy objects like large boxes, pieces of furniture etc. on the seat.

The basket is available as an accessory, it can be taken off the rollator and can be used as a shopping bag or alike, also for bringing light objects like fruit, yarn, a thermos, a book and such.

Maintenance: Clean the rollator with not too hot water and a little detergent. Don't put the rollator in hot places or where it is very damp.

Warranty:

We provide a one year warranty. It does not cover wear parts like wires and wheels.

How to unfold.

(See pictures on page 3!)

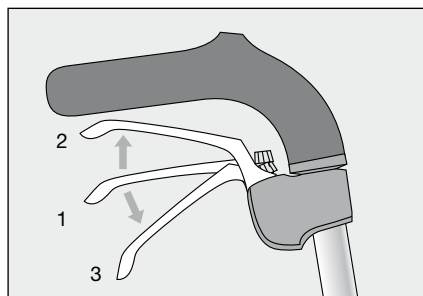
1. Rollator standing. Unfold the rollator by pushing the two sides apart from each other.
2. Make sure that the rollator is fully unfolded and the locking hook under the seat snaps in place.

How to fold.

3. Lift the seat in the middle and the locking hook under the seat is released and the rollator is folded. Press the sides together if needed.


Handle height adjustment.

4. Loosen the two adjustment screws. Adjust the handle height and tighten screws. Follow the markings on the tube.

Brake functions.

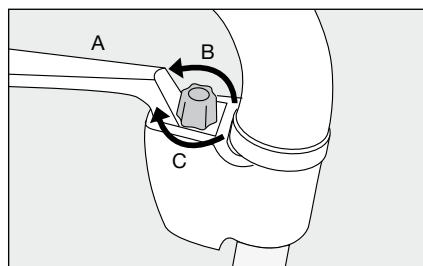1. In position 1 the rollator is rolling freely.
2. To brake the rollator pull the handle up to position 2.
3. For parking brake push handle down to position 3, where the brake locks.

Brake adjustment.

- 1: The brakes are adjusted with the adjustment screw at the brake handle (A).
 2. Adjustment of the wire tension is done by turning the adjustment screw (B-C).
- If the brake is too loose, turn in direction (B).
If the brake is too hard turn in direction (C).

Technical data:

(See pictures on page 2)

Tested according to ISO 11199-2
for a maximum user weight of 130 kg.

Max. width	58.5 cm.
Length	67 cm.
Weight	6.3 kg.
Handle height	adjustable from 77 to 95 cm.
Seat height	59.5 cm.
Front wheel diameter	24.5 cm.
rear wheel diameter	20 cm.
Bag load	max 12 kg.
Rear wheels with soft suspension.	

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Varsam AB