

Mobilex rollator "Leopard"

Item no. 312301, 312302, 312304, 312306, 312308 and 312309

User manual



MOBILEX A/S
Grønlandsvej 5
DK-8660 Skanderborg
Denmark

www.mobilex.dk

1. Introduction

Dear Customer

Thank you for buying a high-quality Mobilex product. This user manual contains a detailed product description and important guidelines for correct use of the product. You must read the instructions carefully before using the product. Please always observe all guidelines, in particular the security instructions, in order to use the product safely.

Intended use of the device

The Mobilex rollator "Leopard" is a medical device which has been designed to increase the mobility of persons who have difficulties standing or walking. The rollator enables those persons to regain the ability to perform daily activities. The rollator is suitable for persons who as a result of various diseases need stable walking support. The rollator should only be used as a support for standing, walking and sitting. Any other use is prohibited.

Indications

The device is indicated for use as walking, standing and seating support by persons who have difficulties standing or walking. The variety of such difficulties is very wide but the symptoms are similar: people who because of their health status have difficulties walking or standing without and aid are indicated to use rollators. The final decision to choose the correct type of device depends on the condition of each individual person and should always be taken by professional physician or a physiotherapist



NB The rollator should not be used by persons with balance problems whose muscles are severely weakened, and thus unable to stand or walk on their own. If in doubt it is recommended to consult a professional physician or a physiotherapist.

If you have any questions, please contact your dealer or Mobilex A/S

Mobilex A/S

Grønlandsvej 5
DK-8660 Skanderborg
Denmark

Phone: +45 87 93 22 20
Telefax: +45 87 93 17 77
Email: info@mobilex.dk

2. Security Instructions

- Use only the product if it is technically flawless
- In case of any product damage or faults please contact your dealer immediately
- Observe the data on the product label
- Use the product only for the described purpose
- Please notice that there is a risk of injury at all joints on the rollator
- Always check before use that the wheel brakes function perfectly
- Use the rollator only when it has been fully unfolded and the locking device has been activated
- The rollator should only be used on a flat stable surface, indoors or outdoors
- The seat may only be used when the brakes are firmly locked
- Both brakes should always be activated or locked simultaneously
- Max. load of the rollator is 150 kg in total
- Max. load of the shopping bag is 5 kg
- It is not allowed to carry an extra person on the seat
- The product must not be changed without the manufacturers written permission
- The rollator must always be braked when not in use

3. Description


The Mobilex rollator "Leopard" has a powder coated frame and 4 wheels. The light-weight frame makes it easy to fold the rollator and offers easy storage and transport. The rollator runs easily and gives a stable support. The height of the push handles can easily be adjusted. The rollator is delivered with a mounted seat, back strap, crutch holder and a shopping bag.

Parts supplied and mounting instructions

The Mobilex rollator "Leopard" is delivered in a carton with the following content:

Parts supplied:

- ✓ 1 rollator inclusive mounted seat, back strap, crutch holder and a shopping bag
- ✓ This user manual

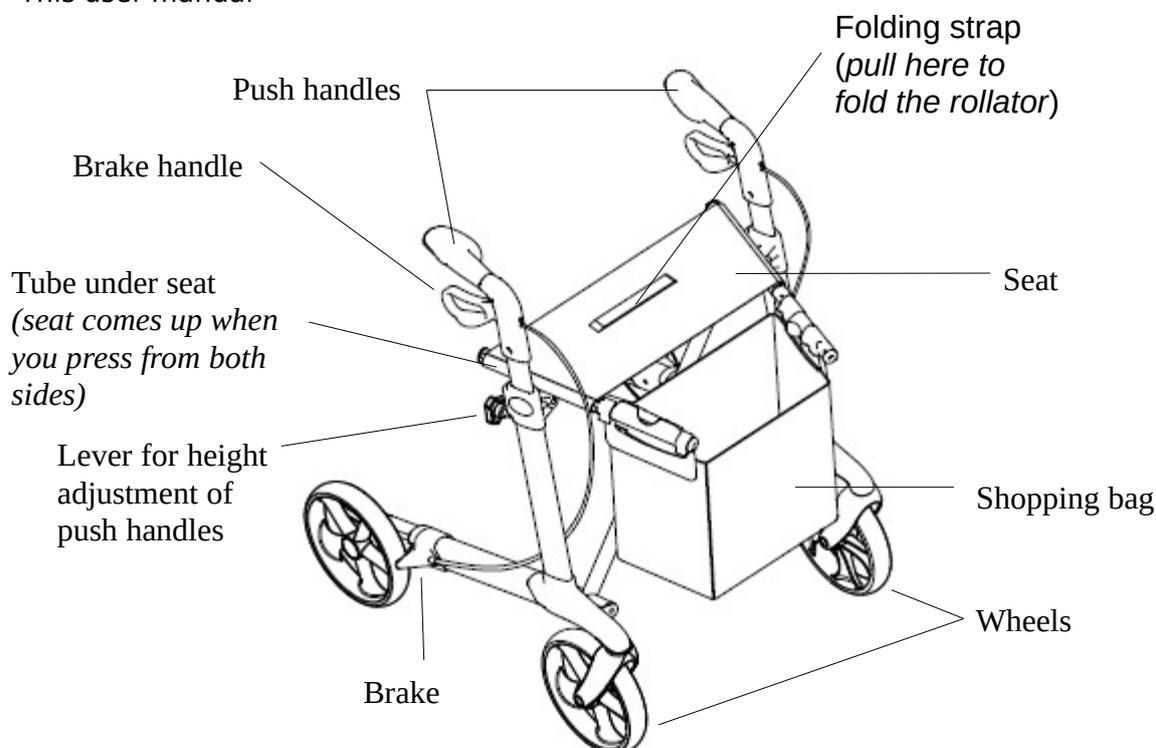

Control upon delivery

After unpacking the rollator please check if all parts shown in the drawing have been delivered. If any parts are missing please contact your dealer immediately.

Unfolding the rollator

- Press down both tubes under the seat until the frame unfolds and you can hear a "click". Now the locking device is in a fixed position.
- To be sure that the frame has been fully unfolded check that it is probably locked.
- Attach the strap which connects both push handles (see photo at the right). When the rollator has been unfolded, you must check that the strap is blocked.
- Check that the levers for height adjustment have been firmly tightened.
- Check that both handles are at the same height and stable.



Use of a defective rollator is strictly forbidden.

Always check before use the function of all parts of the rollator. If the above recommendations are not followed, injuries may occur.

Folding the rollator

- Pull the strap at the seat until the frame folds together.
- Now the rollator is folded together.

Height adjustment of push handles

- Unfold the rollator

- Loosen the lever by turning it against the clock.
- Adjust the height
- Turn the lever to clockwise
- Make sure both push handles are adjusted in the same height.

Brake function

- To brake you must pull both brake handles towards yourself
- The more you pull the higher the braking capacity
- To ensure that the rollator remains STOPPED you must keep the brake handles pulled
- To make the rollator MOVE you must let go of the brake handles whereby the braking will stop



Never release the brakes when seated on the rollator!

Always activate the brakes when not using the rollator!

When placing the rollator on a non-plane surface you must always activate the brakes!

Activating and deactivating brakes

- To stop the rollator pull the brake handles upwards
- To park the rollator push the brake handles downwards until a click is heard
- Do not sit on the rollator unless the brakes are in a locked position
- Before setting the rollator into motion you must stand up in the walking direction and release the brake handles by pulling them upwards



The brakes must always be in a locked position when the rollator is parked!

Never use the seat if the brakes are not locked! *When seated, never lean backwards.*

It is recommended to sit only at the front part of the seat.

Function description

As the frame can be closed the rollator can fold, and as a result it requires very little space. Push handles can be freely adjusted to fit the size of your body. The seat allows you to rest.

Accessories

For the Mobilex "Leopard" rollator various accessories are available such as crutch holders, speed reducing brake etc. Please contact your dealer.

Transport recommendations

It is recommended to transport the rollator in the folded position. Its construction allows easy unfolding and folding. You should check that the rollator is in a stable position during transport to avoid any deformation.

Use


Stand in the middle behind the back wheels. Grab the push-handles and smoothly start walking. Due to the special design of the brakes, only a minor effort is required to use the brakes.

4. Technical data

Moblex rollator Leopard	Standard 312301 (red) 312304 (grey) 312309 (blue)	Std. + softwheels 312308 (red) 312306 (grey)	Small 312302 (grey)
Height of the push handles (adjustable)	74 - 102 cm	74 - 102 cm	66 - 86 cm
Seat height	62 cm	62 cm	55 cm
Total width opened / folded	60 / 22,5 cm	60 / 22,5 cm	60 / 22,5 cm
Inner width between push handles	46 cm	46 cm	46 cm
Length (depth)	66 cm	66 cm	66 cm
Weight (+ bag 380 g)	7,4 kg	7,6 kg	7,1 kg
Max. load (rollator / bag)	150 kg / 5 kg	150 kg / 5 kg	150 kg / 5 kg
Brakes	2 lockable with parking function		
Wheel sizes	Front and back 200 x 35 mm		

Product label

A product label is placed on the lower frame

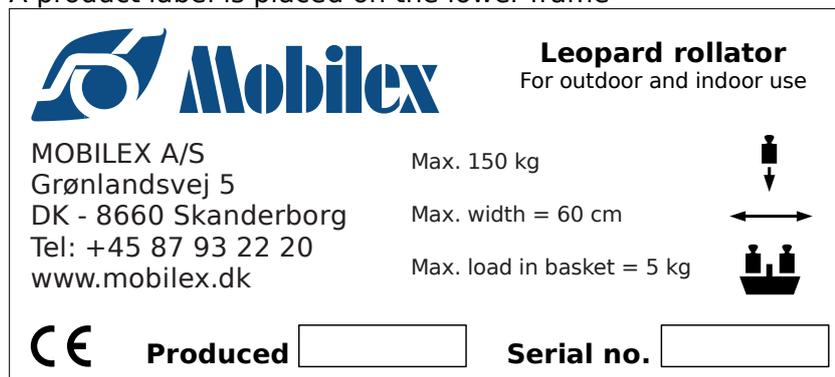

Example of product label (not original)

5. Maintenance and care

Normal dirt on metal and plastic parts can be removed with normal cleaning agents. Check the contents of the cleaning agents and use only products suitable for cleaning and disinfection. Do not use solvents or abrasive cleaning agents

Despite the solid structure and use of resistant and anti-corrosive materials the rollator is subject to wear. It is therefore recommended to check the rollator regularly (once a year as a minimum). Such a check should comprise: A visual control of all parts of the frame: if they have been bent, damaged, worn or corroded and in particular a control of the following

- ✓ Rollator frame
- ✓ Push bars, handles and strap
- ✓ Brake handles and brake cables
- ✓ Forks and front wheels
- ✓ Braked back wheels
- ✓ Seat and locking device
- ✓ Shopping bag
- ✓ Check if any mounting parts are missing (e.g. nuts, screws, rivets ...) and if they function and are firmly tightened
- ✓ Check the function of the brakes
 - by lowering the speed
 - by braking the wheels

Cleaning the wheels

It is recommended to clean the wheels in hot water with a mild cleaning agent. Lubricate the bearings with oil periodically or when the wheels start squeaking.

6. Reuse

The rollator is suitable for reuse. Prior to reuse the rollator must be cleaned and disinfected. Furthermore, it must be checked in every respect mentioned under "5. Maintenance and Care".

7. Troubleshooting

Problems	Possible Causes	Solutions
The rollator is difficult to guide or push	Dirt between wheels and forks	Remove dirt or hair between wheels and forks
The brakes do not work or work only in one side	The brake shoes are worn, or the brakes are not adjusted correctly	Adjust the brakes. Replace the brake shoes if needed (→ service centre)
The seat does not fit when the rollator is unfolded	The locking device is in a wrong position or defective	Check that the locking device is in a correct position. Replace it if needed (→ service centre)

8. Disposal of the product

The product may not be disposed of with household waste but must be brought to the local recycling site.

9. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

10. Declaration of Conformity (CE)

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark, hereby declares that the

Mobilex Rollator "Leopard"

Item no. 312301, 312302, 312304, 312306, 312308 and 312309

is designed and produced in accordance with the technical documentation and the basic requirements of the:



- **EU Council Directive 93/42/EEC for Medical Equipment**

Lars Nygaard
Managing Director
Mobilex A/S

Skanderborg, 03.08.2018



.....
Signature