**TOPRO
TROJA CLASSIC**

TOPRO

NO	Bruksanvisning	3–6
EN	User manual	7–10
DE	Bedienungsanleitung	11–14
FR	Manuel d'utilisation	15–18
NL	Gebruiksaanwijzing	19–22

www.topro.no

Made in Norway

CE


A 814726 M✓ S✓ X✓	B 814728 M✓ S✓	C 814059 M✓ S✓
		
D 814625 M✓ S✓	E 814622 814672 814643 814641 814677 M✓ S✓	F 814046 M✓ S✓
		
G 814042 M✓ S✓ X✓	H 814025 M✓ S✓ X✓	I 814043 M✓ S✓ X✓
		
J 814141 M✓ S✓ X✓	K 814024 M✓ S✓ X✓	L 814023 M✓ S✓ X✓
		
M 814068 M✓ S✓ X✓	N 814608 M✓ S✓ X✓	O 814624 M✓ S✓ X✓
		
P 814032 M✓ S✓ X✓	Q 814029 M 814035 L	R 814026 M✓ S✓ X✓
		
S 814014 M✓ S✓ X✓	T 814009 M✓ S✓ X✓	U 814829 M✓ S✓ X✓ 814845
		

TOPRO TROJA M
TOPRO TROJA S
TOPRO TROJA X


814750
814770
814740


1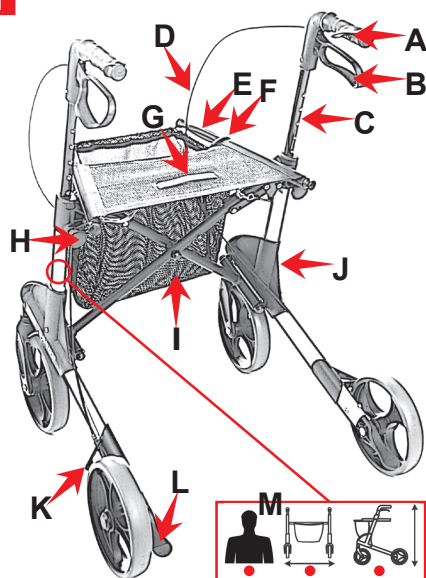


2

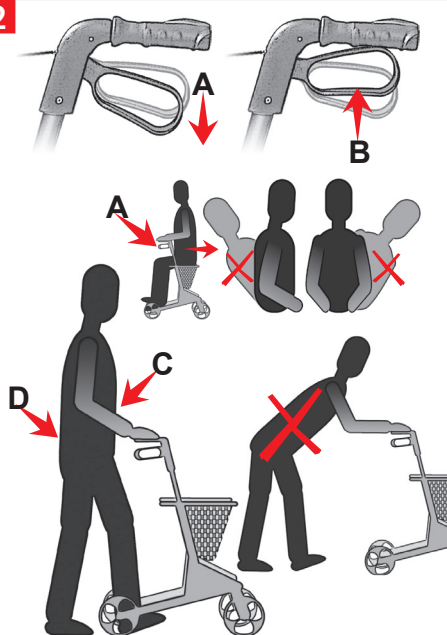
3

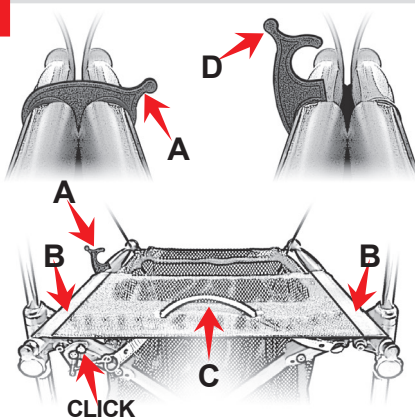
4

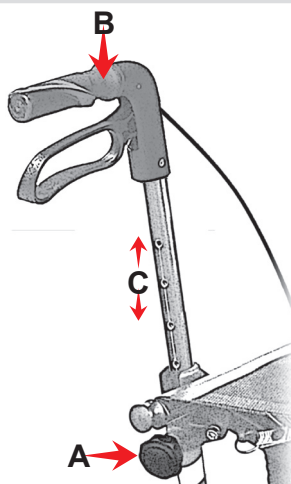
5

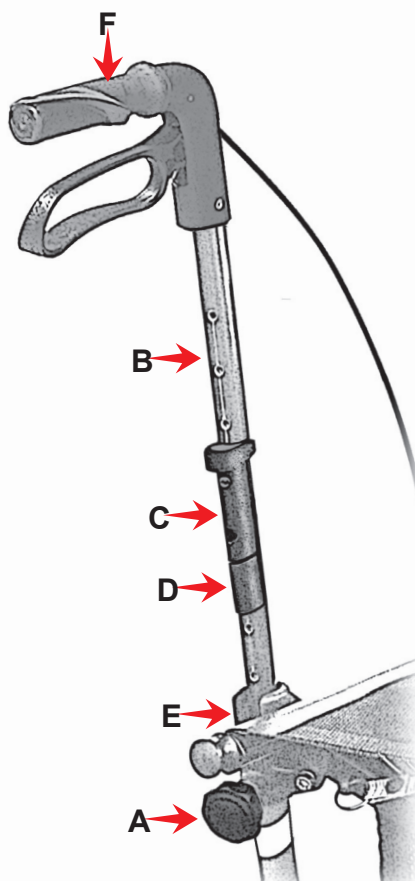
6

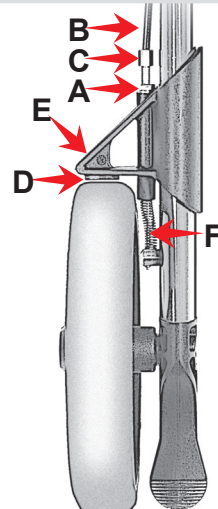
7

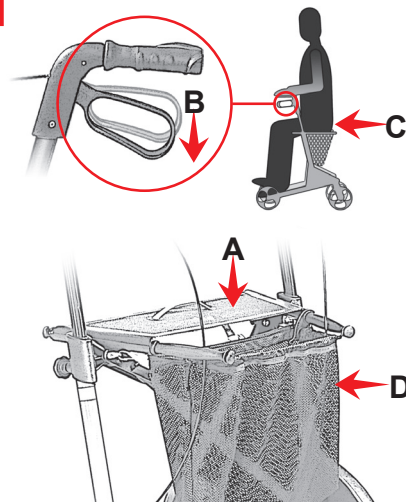8

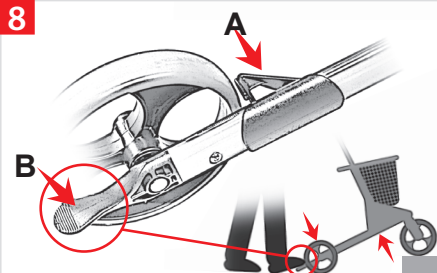9

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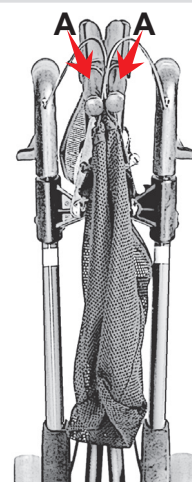Welcome as a user of a TOPRO PRODUCT

Every TOPRO product is developed, designed and manufactured in Norway. Due to advanced manufacturing by using the best equipment, we are able to offer great accuracy and maximum quality in all levels of our production. This product conforms to the Medical Device Directive 93/42/EEC. It is tested and approved according to ISO 11199-2:2005.

On receipt of the goods, please check your product. If you have any queries please contact your mobility shop or dealer immediately. The same applies if you have general questions about the product. If you have problems reading this user manual, an electronic version is available on www.topro.co.uk

Warranty // Expected life span of the product

TOPRO TROJA is guaranteed free of faults and defects for a period of 7 years. Damage caused by incorrect use or parts that are exposed to natural wear and tear (e.g. brake block, brake cable, wheel, seat, basket and handgrip) are exempt from the 7-year guarantee. For repairs during the warranty period, please contact your local mobility shop or dealer.

The warranty is void if unauthorised spare parts and accessories have been or are being used on the product.

The expected life span of the product is estimated to be 10 years, provided that it is used in the correct manner according to this user manual and its safety and maintenance instructions.

The red numbered squares below refer to the numbered illustrations on the inside of the cover.

Get to know your TOPRO PRODUCT 1

- | | | |
|-------------------------|--------------------------------------|-----------------------------------|
| A Handlebar | E Seat bars/carrying handle | I Basket |
| B Brake handle | F Lock clamp | J Side frame |
| C Handlebar tube | G Seat with strap for folding | K Brake block / Edge guard |
| D Brake wire | H Lock wheel for the handle | L Tilt function |
- M** The label on the rear side of the left tube shows the product name/model and
- Maximum weight of user ■ Maximum width of rollator ■ Maximum height of rollator
- A separate label on the right tube indicates the serial number (SN) and production year and month.

Measurements	TOPRO TROJA M	TOPRO TROJA S	TOPRO TROJA X
Recommended user height	150–200 cm	135–170 cm	125–160 cm
Maximum weight of user	165 kg	125 kg	80 kg
Maximum capacity basket	10 kg / 20 l	10 kg / 20 l	5 kg / 11 l
Handle height	780–1000 mm	670–860 mm	635–790 mm
Maximum length	650 mm	650 mm	650 mm
Maximum width	600 mm	600 mm	520 mm
Width between handles	470 mm	470 mm	370 mm
Seat height	620 mm	540 mm	500 mm
Turning diameter	840 mm	840 mm	790 mm
Diameter wheels	200 mm	200 mm	200 mm
Width wheels	36 mm	36 mm	36 mm
Handles	Anatomic	Anatomic	Anatomic
Grip distance driving brake	65 mm	65 mm	65 mm
Height when folded	800 mm	740 mm	650 mm
Width when folded	240 mm	240 mm	240 mm
Length when folded	650 mm	650 mm	650 mm
Product weight	6.8 kg	6.6 kg	6.1 kg

Intention

The TOPRO TROJA shall give support to users with reduced balance and/or reduced walking ability. It is developed to be pushed, not pulled. The integrated seat provides an opportunity for the user to rest if needed. The fabric basket is designed for carrying items weighing up to a total of 10 kg (model M and S) / 5 kg (model X). The product is not suitable for persons with low strength in arms, with very poor balance, or with considerable cognitive disabilities. The product is designed and approved for both indoor and outdoor use. Outdoor the rollator is intended to be an aid for walking on roads and paths. It is not suitable in rough and/or steep terrain or steep inclinations, for example stairs.

Important to know about safety before use

- Take care when folding and unfolding, do not trap your fingers.
- Check the brakes every time before use and that the rollator locks into position when opened.
- Not to be used by persons heavier than the maximum user weight (see table).
- Do not use to transport heavy loads or persons.
- The rollator is not a toy and therefore should be treated in an appropriate manner.
- Make sure that the lock wheel for height adjustment for the handles is locked tightly.
- Be aware that some parts of the rollator might feel hot or cold when exposed to extreme temperatures.
- Do not alter the product as this might put your safety at risk and the guarantee will not be void.

Important to know about safety when in use **2**

- When stationary or parked, always use the parking brakes on both sides **A**
- Before sitting on the seat, make sure the parking brakes on both sides are in the locked position **A**
- Do not lean or bend sideways or over the back of the rollator whilst in the seated position. Do not tilt the rollator more than max. 5° sideways, to avoid risk of falling. Due to the same reason bags etc. must never be hung on the handlebars.
- The driving brakes can be held in a little **B** to reduce speed, but must not be in locked position when the rollator is in motion.
- To give the right support and safety, make sure the rollator is kept as close to your body as possible **C**
- Walk straight, look forward and hold your backside in **D**. That is the easiest way to keep your balance.
- Do not push the rollator too far from your body as this can cause unnecessary strain and you could fall.
- Be aware that the rollator may roll faster than you walking downhill, which may put you in risk of falling.
- Do not use the rollator on stairs. Be careful when descending kerbs.
- Be careful of flared skirts and trousers as they could be caught between the brake block and wheel, which could lead to falling. We therefore recommend not wearing flared/wide clothing.
- For safety reason it is prohibited to sit on the rollator during public transportation. Place your rollator securely and use the parking brake.

Assembling / adjusting

TOPRO TROJA is delivered assembled. Before use, you must unfold the rollator. The height of the handlebar tubes will then need adjusting to suit the user's needs.

Unfolding / folding **3**

Unfolding: Loosen the lock clamp that is over the seat bars **A**. Press the seat bars down and out **B**. The side frames will slide apart. Press down the seat bars. You will hear a clicking sound. The rollator is now locked in its open position.

Folding: Pull the strap on the seat **C** and press the side frames together. Lock the clamp **D** over the seat bars.

Lowering and height adjusting of the handles **4**

Lowering: Unscrew the lock wheel for the brake handles **A** (approx. 4–5 turns). Push the handlebar tubes down until they stop **B**. Tighten the lock **A** again.

Height adjusting: Unscrew the lock for the brake handles **A** (approx. 4–5 turns). Adjust the handlebar tubes to a suitable height then pull them slightly up and down **C** until you hear a clicking sound. Tighten the lock **A** again. Adjust both handlebars to the same height. *It is recommended that the handlebars are set to the same height as your wrists when your arms are hanging alongside your body.*

Height adjustment using the memory function **5**

You have the opportunity to set the height of the handles at a standard setting fitted to your requirements so that you easily can find the same height again. This can be necessary when the rollator is being transported or stored as compact as possible. Decide the height of the handlebars. Count visible holes at the back of the handlebar tubes. Unscrew the lock wheel **A**. Pull the tube **B** out of the frame. The attached part **C** will now follow the handlebar tube. Move the attached part **C** on the tube **B** so the amount of holes corresponds to the amount counted earlier. Fit the memory chip **D** on the handlebar tube in the first available hole under the attached part **C**. Push the handlebar tube **B** down and into the side frame **E** until the attached part is locked. Screw the lock wheel **A** half in. Pull the handlebars **F** up until they stop and then tighten the lock wheel **A** properly. From now on, when the handlebars are in the lowest position, they will stop in the preset position automatically when pulled up.

Adjusting / checking and maintaining the brakes 6

Adjusting / checking: Loosen the nut **A** beside the brake. Tighten the brake wire **B** with the adjustment screw **C**. Adjust both brake blocks so the distance to the wheel surface is approximately 1 mm **D**. Make sure the brakes are not adjusted too tight and pressing hard into the rubber when the parking brake is on.

Maintaining: Clean the brake block **E** and the brake spring **F** frequently. The wheels can be replaced if they are worn or damaged.

Using the brakes 2

Make sure the brakes work before each trip. Each brake handle **A** works on each of the rear wheels.

The rollator wheels are made of a special soft rubber, which gives an optimal braking capacity on slippery floor.

Wrong use can cause unsafe situations and damage to the rollator: *The rollator must not be driven or pushed with force when the parking brakes are on or when the wheels are partly locked using the brakes. This indicates incorrect use and can cause wear and tear that makes the tires and brake quality weaker.*

Parking brakes: Push both brake handles **A** down until they lock. Both rear wheels are now locked. Squeeze both brake handles upwards to release the parking brakes.

Driving brakes: Pull both brake handles **B** upwards to reduce speed. *Do not use continuously while the rollator is in motion.*

Seat // Basket 7

The parking brakes must be on before using the seat **A**. Push both brake handles **B** downwards until they lock.

Both rear wheels are now locked. Sit with your back towards the basket **C**. The basket **D** can be removed and washed gently at 40° C.

Edge guard // Tilt function 8

Edge guard: The brake blocks **A**, which are in front of the rear wheels prevent the rear wheels from catching on corners, door frames etc.

Tilt function: To use the tilt function use the front of the foot and press down on the tilt pedal **B**, which is beside the rear wheel. This helps to tilt the rollator over thresholds, edges etc. **Please note:** Take care when descending kerbs. Do not use the rollator in stairs.

Carrying the rollator 9

The rollator must not be carried by the seat strap or the brake wires. Fold the rollator together as shown on the illustration. Carry the rollator by the folded seat bars **A**.

Transport

The rollator should be transported in the folded and locked position (see paragraph *Unfolding / Folding*). Ensure that the rollator cannot unfold unintentionally. Be careful when loading the rollator into a vehicle, giving attention to securing of the item.

Storage

The rollator should be stored in an upright position. It can be stored in- and outdoors, if stored outdoors then under cover.

Material // Recycling

The rollator is made of plastic coated aluminium pipes, plastic couplings and material of plastic and polyester. Most parts can be recycled. Dispose the rollator according to the applicable regulations in your country. For information, please contact your public authorities.

Cleaning

The rollator can be steamed, hosed and washed with normal household detergents. Do not use scrub detergents, steam- or high-pressure cleaners.

Disinfection

Disinfection must only be carried out by authorized personnel and by wearing sufficient protective equipment. The surface must be cleaned with a disinfection that contains 70–80 % ethanol. We advise against disinfection containing chlorine and phenol. *The manufacturer cannot be held responsible for any damage or wounds that might occur by the use of harmful disinfection or disinfection carried out by unauthorized personnel.*

Inspection // Maintenance // Re-use

It is recommended that general inspection and maintenance is carried out and that the items below are checked on a regular basis (how frequent is determined by how and how often the rollator is used): Tubes, frame and screws, handlebars and handlebar tubes, brakes, wheels, seat and accessories. This also applies when the rollator is to be prepared for re-use. Please refer to the detailed instructions presented in this user manual about maintenance of the brakes and cleaning and disinfection of the rollator.

If the rollator does not work properly

Make sure that the rollator is unfolded correctly and that both brake handles are in expected position.

Accessories

For pictures and TOPRO article numbers please see the inside of the cover of this user manual. Accessories mounted on the rollator may influence stability, we therefore advise you to use them carefully.

- | | |
|--|--|
| A Back support with padding | L Lock |
| B Tray | M Umbrella with attachment (black) |
| C Anti slip mat for tray | N LED lamp |
| D Net basket, detachable | O Reflective panels |
| E Shopping bag, detachable | P Drag brake (pair) |
| F Rear bag with zipper | Q Ice spurs for shoes size M and L |
| G Transport bag | R One-handed brake |
| H Crutch holder | S Guiding handle for e.g. visually impaired |
| I Bottle holder | T Basket for oxygen bottle (compressed, not liquid) |
| J Flower stickers, set with four stickers | U Forearm supports (for indoor use on plain foundation) |
| K Nameplate | |

The accessories are to be ordered separately to give the possibility to compose a TOPRO TROJA that suits the individual users need. Contact your mobility shop, dealer or TOPRO for an updated overview of the accessories, or visit our homepage www.topro.co.uk

In case of questions or product faults, please contact your mobility shop or dealer.

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