

# Canopy Fitting Instructions

## Summer Canopy No 1

### Fitting to Arm Rests

#### PLEASE READ INSTRUCTIONS IN FULL BEFORE FITTING

For seat arm rests that are Steel Box section and adjustable in and out from the side of the seat (fig. A)

Place a rubber pad in each of the Top Hat sections (fig. B) (or fig. B1) for armrests with a round bar; place these over the arm rest on the inside facing outwards with the longest leg of the top hat (fig. B) at the rear towards fig. C1) - Bolt together with the nuts and bolts supplied just enough to hold firm DO NOT TIGHTEN. Bolt (fig. D1) to (fig. C1) with the nuts and bolts supplied - just enough to hold firm DO NOT TIGHTEN. Place the two poles on (fig. D) then the top section on top of the poles - now adjust height and angle. Move (fig. B and fig. C) up and down the arm rest for height and the swivels (fig. C2) to adjust the angle of the top, when satisfied tighten all the nuts and bolts in this section. Adjust (fig. D1 and fig. C1) forward or back as required for length and tighten nuts and bolts. If extra length is needed adjust the slots in (fig. C) accordingly.

Make sure that all nuts and bolts are secure, also the screw handle of Allen screw that holds the arm rest in place is tightened.

Lay the canopy over the frame and connect the Velcro strips to the front and sides; turn the sides of the top upwards and connect the middle of the window to the top (fig. E) by folding the window flat over the bar and connect the Velcro. Connect the side windows to the Velcro on the top canopy; replace the sides.

To fold the window back to the rear - turn the sides of the top canopy up and pull the window off the top and roll INWARDS to the back - when fully wound connect the two Velcro tabs to the top rear of the canopy frame, now fold down the two sides of the canopy top and over the window leaving the back connected to the top (fig. E).

THE TWO STRAPS AT THE BACK OF THE CANOPY TOP MUST BE FITTED AT ALL TIMES TO THE BASE OF THE FRAME (fig. D2) WHETHER THE SIDE WINDOW IS FITTED OR NOT.

Fit the coachman's cover pouch to the rear (fig. D2) - when the coachman's cover is required simply remove from the pouch, place over the knees and take the two straps around the waist and secure at the front with Velcro fastening.

CAUTION TO BE TAKEN IN WINDY CONDITIONS

Summer Canopy No. 1

