


Horizontal position of headrest should be set for occupant comfort within 10mm of head. Height of headrest should be set to centre of occupant's head. The height of the headrest is set by means of a fixing knob as shown in the diagram. Release of this knob allows the cushion tube to slide up or down. When at the required height the knob should be set finger-tight. Do not overtighten.


Attachments

- Instructions
- M5 bolt & nut
- M5 (5m) wrench
- M3 (3m) wrench
- Fixing knob
- Packing

Assembly figure

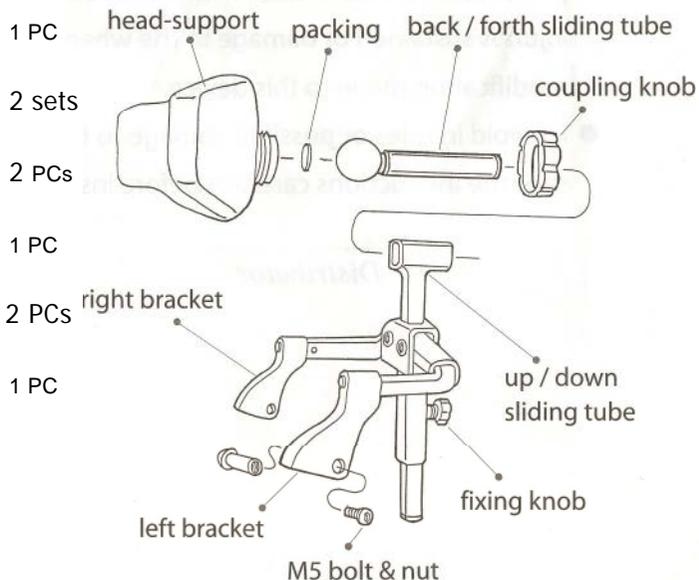

How to install:

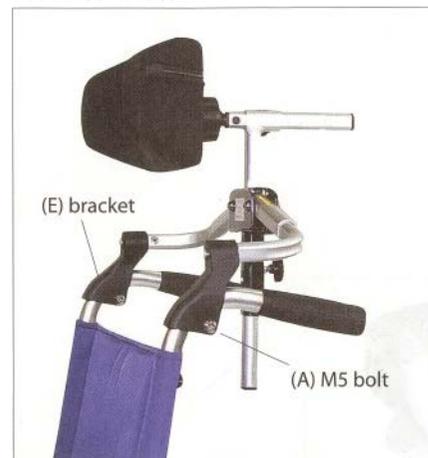

Unfold wheelchair first, loosen bolts (B) to allow inner tubes to slide freely. Fit brackets (E) onto the curved part of push-handles, then tighten bolts (A)



Have a person help by slightly pushing rear handles of wheelchair apart, simultaneously adjust center joints as high as possible (in order to obtain adequate support tension for headrest) then tighten bolts (B)

How to use:



When using wheelchair, unfold wheelchair first, then prop up the headrest.



Conversely when folding wheelchair, first pull down the headrest to release the support tension, then fold wheelchair.

NOTE: DO NOT put fingers into the joints of the headrest.