
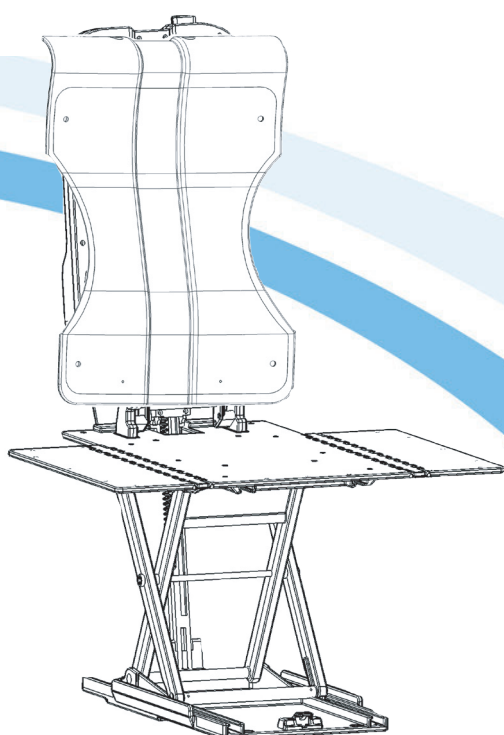
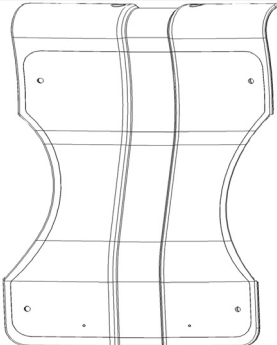
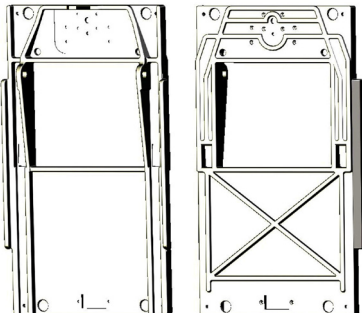
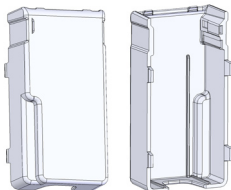
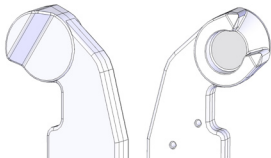
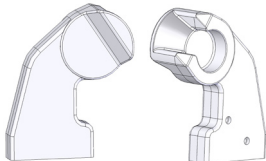
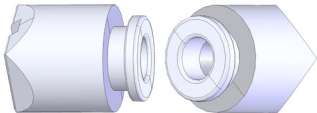
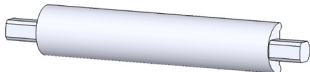
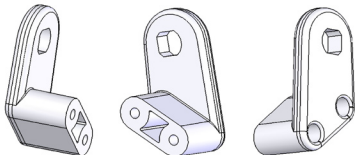
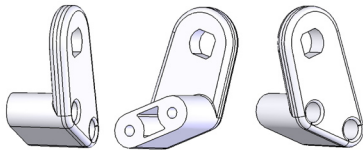
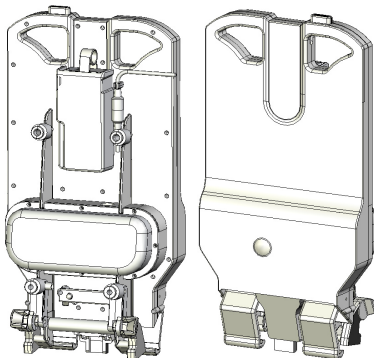
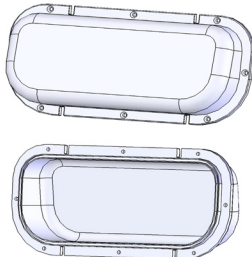
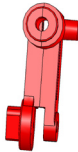
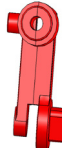






drive






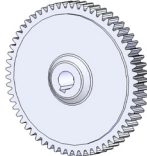







DRIVE MEDICAL
SPARE PARTS & ACCESORIES
BATHLIFT Riviera (2+3)



Spare part	Product description	Picture no. (P)/ spare part no.	Product code
	Backrest plate - ergonomic	P2 / 79	400 631
	Base plate	P1+P2 / 12	400 424
	Battery compartment	P1+P2 / 2	400 608
	Bearing pedestal, left	P1+P2 / 58	400 415
	Bearing pedestal, right	P1+P2 / 59	400 414
	Bracket for backrest plate, top (right=left)	P1+P2 / 60	400 591
	Carrier handle	P1+P2 / 96	400 477
	Carrier handle holder, left Holding device, left for carrier handle	P1+P2 / 98	400 476


Product description	Picture no. (P)/ spare part no.	Product code	Spare part
Carrier handle holder, right Holding device, right for carrier handle	P1+P2 / 97	400 475	
Center axis for shear	P1+P2 / 64	400 537	
Drive unit (complete) without backrest plate	5	400 628	
Engine bonnet	P1+P2 / 65	400 590	
Engine bonnet seal	P2+P3 / 18	400 592	
Fitting key holder, left	P1+P2 / 74	400 598	
Fitting key holder, right	P1+P2 / 75	400 599	
Gear wheel M 2,5 incl. shaft (drive shaft)	P3 / 99	400 601	
Gear wheel Z60	P3 / 101	400 586	

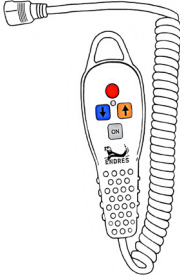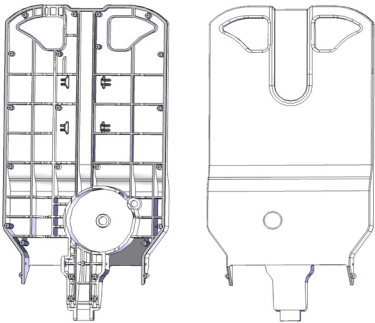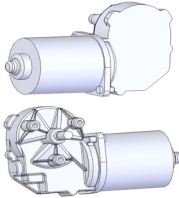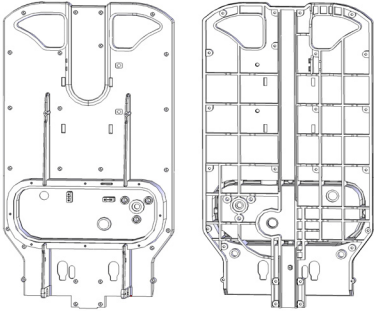Spare part	Product description	Picture no. (P)/ spare part no.	Product code
	Hand control	P1+P2 / 49	400 600
	Hinge case	P1+P2 / 86	400 607
	Insert thread	P1+P2 / 40	400 363
	Lift bar	P1+P2 / 104	400 609
	Motor (Engine)	P2 / 7	400 568
	Motor case	P1+P2 / 68	400 606

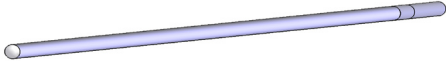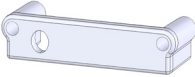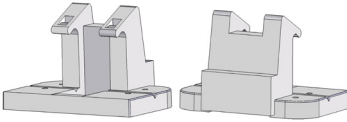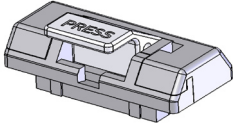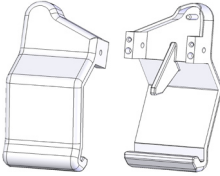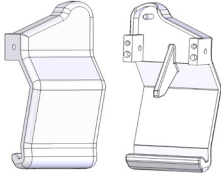Product description	Picture no. (P)/ spare part no.	Product code	Spare part
Pin (right=left) for double hinge	P1+P2 / 90	400 452	
Double hinge (right=left)	P1+P2 / 89	400 451	
Plug D8	P1+P2 / 15	400 468	
Pressure rod	P1+P2+P3 / 24	400 612	
Pressure rod bridge	P1+P2 / 13	411 545	
Ramp, Unlocking device	P1+P3 / 78	400 610	
Release press for central sucker, Holding device for central suction cup	P1+P2 / 106	400 397	
Release & Retaining flap, left	P1+P2+P3 / 53	400 611	
Release & Retaining flap, right	P3 / 54	400 614	

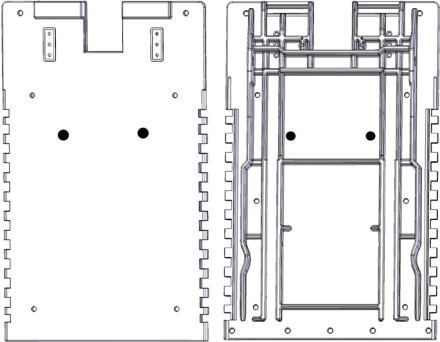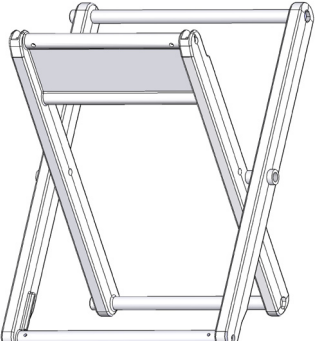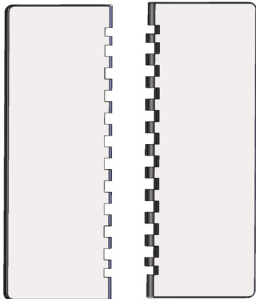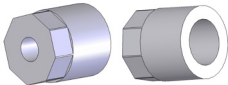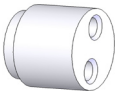Spare part	Product description	Picture no. (P)/ spare part no.	Product code
	Seat plate	P1+P2 / 95	400 575
	Shear	P1+P2 / 87	400 417
	Side flap (right=left)	P1+P2 / 92	400 447
	Slide shoe, Guiding shoe, down (right=left)	P1 / 44	400 445
	Slide shoe, Guiding shoe, top (right=left)	P2 / 43	400 630
	Spigot, Plug for Shear	P1+P2 / 105	400 461
	Thread plug for shear	P1+P2 / 42	400 466

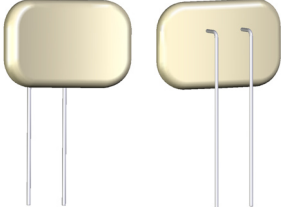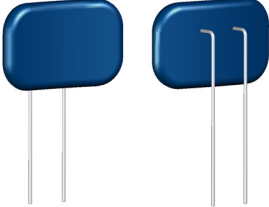
Product description	Picture no. (P)/ spare part no.	Product code	Spare part
Battery	P1+P2 / 1	400 351	
Battery housing	35	400 563	
Charger, black	57	400UK350	
Height adapter 20 mm (set of 6 each)	122	401 536	
Height adapter 40 mm (set of 6 each)	123	401 542	
Height adapter 60 mm (set of 6 each)	124	401 552	
Headrest, white	125	410 200 252	
Headrest, blue	126	410 200 312	

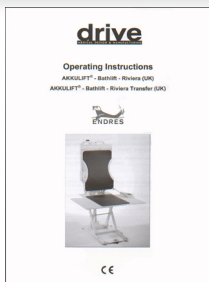Spare part	Product description	Picture no. (P)/ spare part no.	Product code
	Operating instructions Bathlift - Riviera (UK)	32	BAAK0026
	Paperbord container - Box Riviera	52	400 990
	Plug & Washer for covers	131/133	415 252 415 312
	Side flap protector	88	400 570
	Sucker, Suction cup Ø 50 mm, transparent	P1 / 82	400 855
	Sucker grey, Central Suction cup Ø 75 mm, grey	P1 / 83	460 900 213
	Suction cup, set	84	400 856
	Vitaknapp K, white, (rotating aid)	119	102 385
	Vitaknapp L, white, (rotating & transfer aid)	120	101 385


Product description	Picture no. (P)/ spare part no.	Product code	Spare part
Riviera Classic cover set white	109	410 000 252	
Riviera Classic cover set blue	110	410 000 312	
Riviera Comfort cover set white	111	410 100 252	
Riviera Comfort cover set blue	112	410 100 312	
Vitaturnt insert, white	121	112 252	

Technical Data


A	Overall length (backrest in reclined position)	ca. 860 mm
B	Overall width (seat incl. side flaps)	ca. 700 mm
C	Width of seat (seat without side flaps)	ca. 370 mm
D	Width of side flap	ca. 175 mm
E	Depth of seat	ca. 450 mm
F	Overall length of seat	ca. 580 mm
G	Sitting height min.	approx. 80 mm
H	Sitting height max.	approx. 420 mm
I	Height of backrest	approx. 560 mm
J	Width of backrest	approx. 320 mm
K	Angle of inclination (backrest)	12° - max. 40°
L	Width of base plate	approx. 310 mm
M	Length of base plate	approx. 545 mm
	Load max.	130 kg (From SN 306212 max. load 140 kg!)
	Weight, incl. storage battery	approx. 14,8 kg
	Motor unit (without battery)	approx. 7.0 kg
	Seat unit	approx. 7,7 kg
	Duration of lifting/lowering movement	approx. 25 secondes (each)
	Operating voltage	12 V 
	Storage battery	12 V  2 - 2.6 Ah (rechargeable)

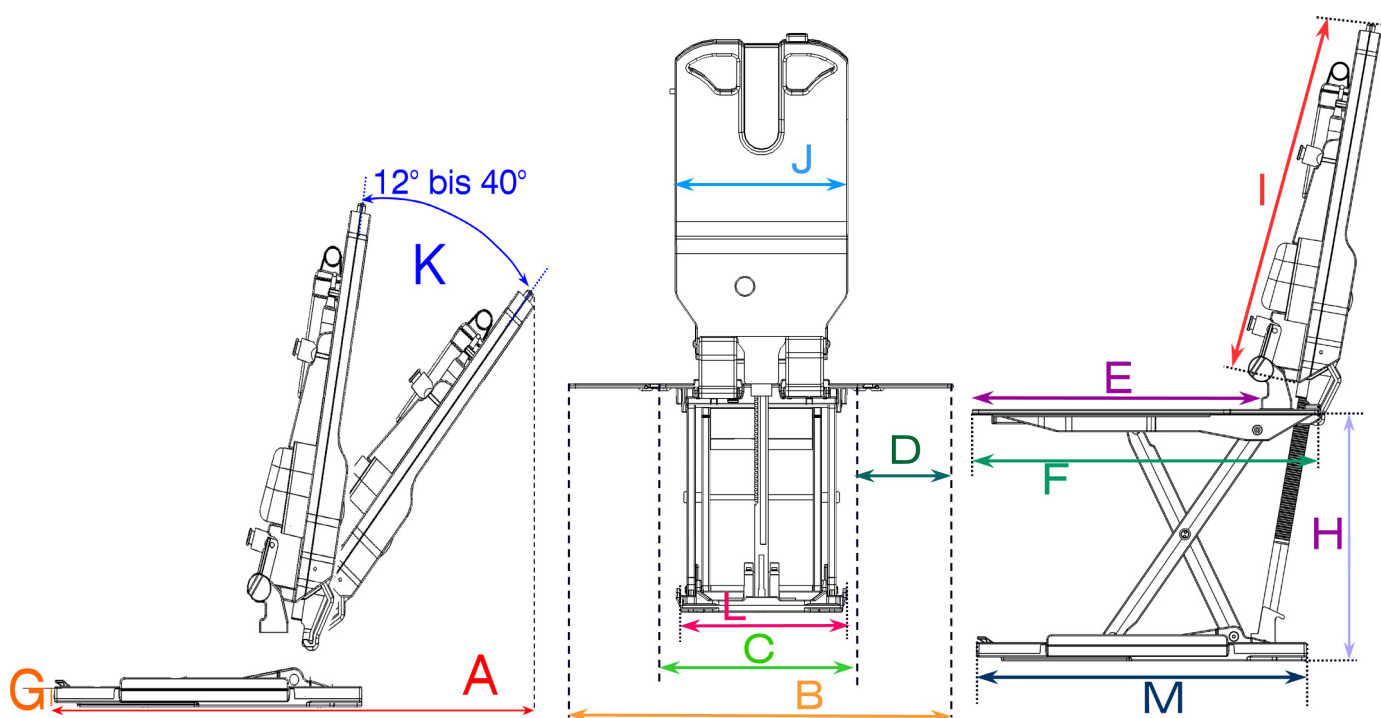


Picture 1 (P1)

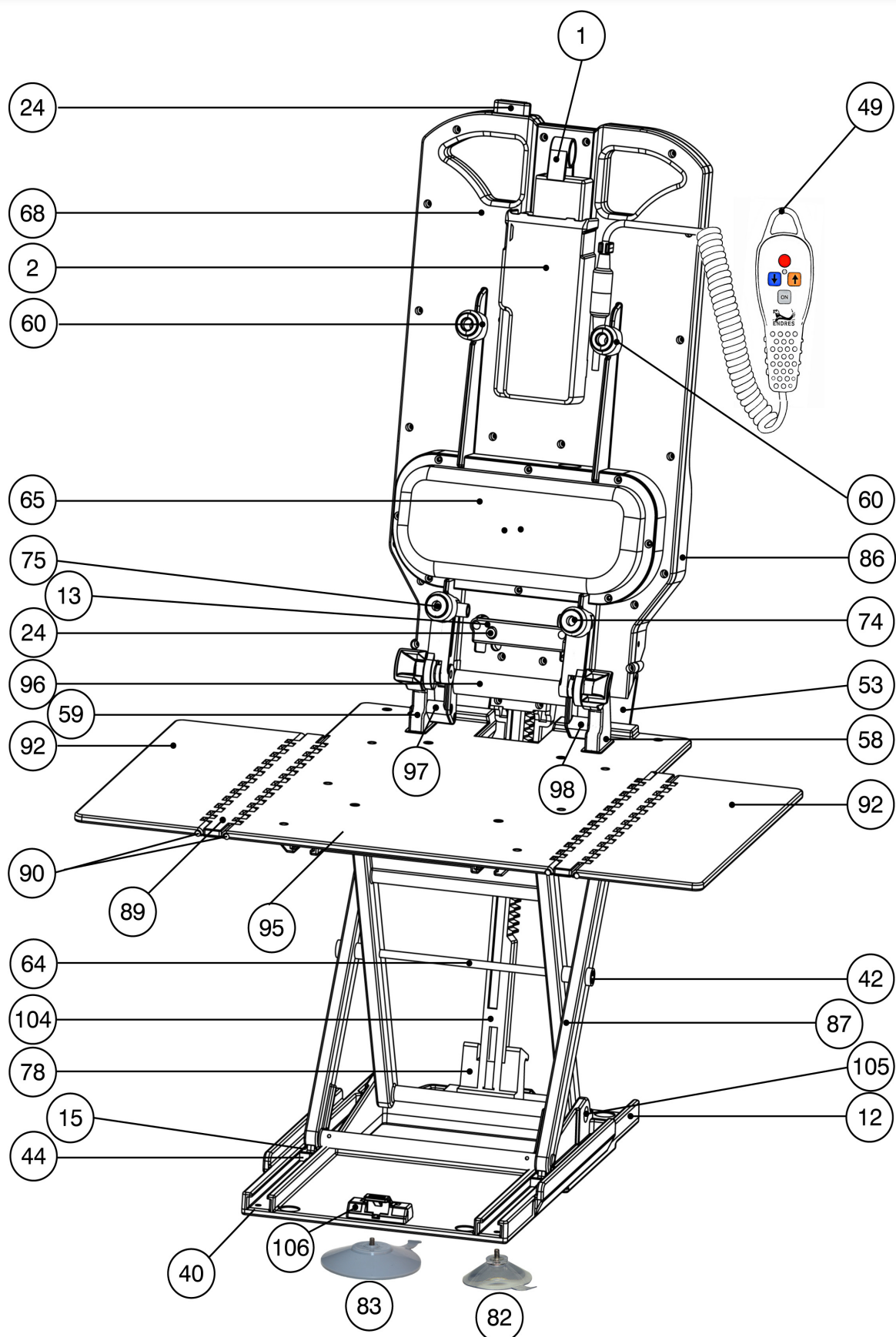


Picture 2 (P2)

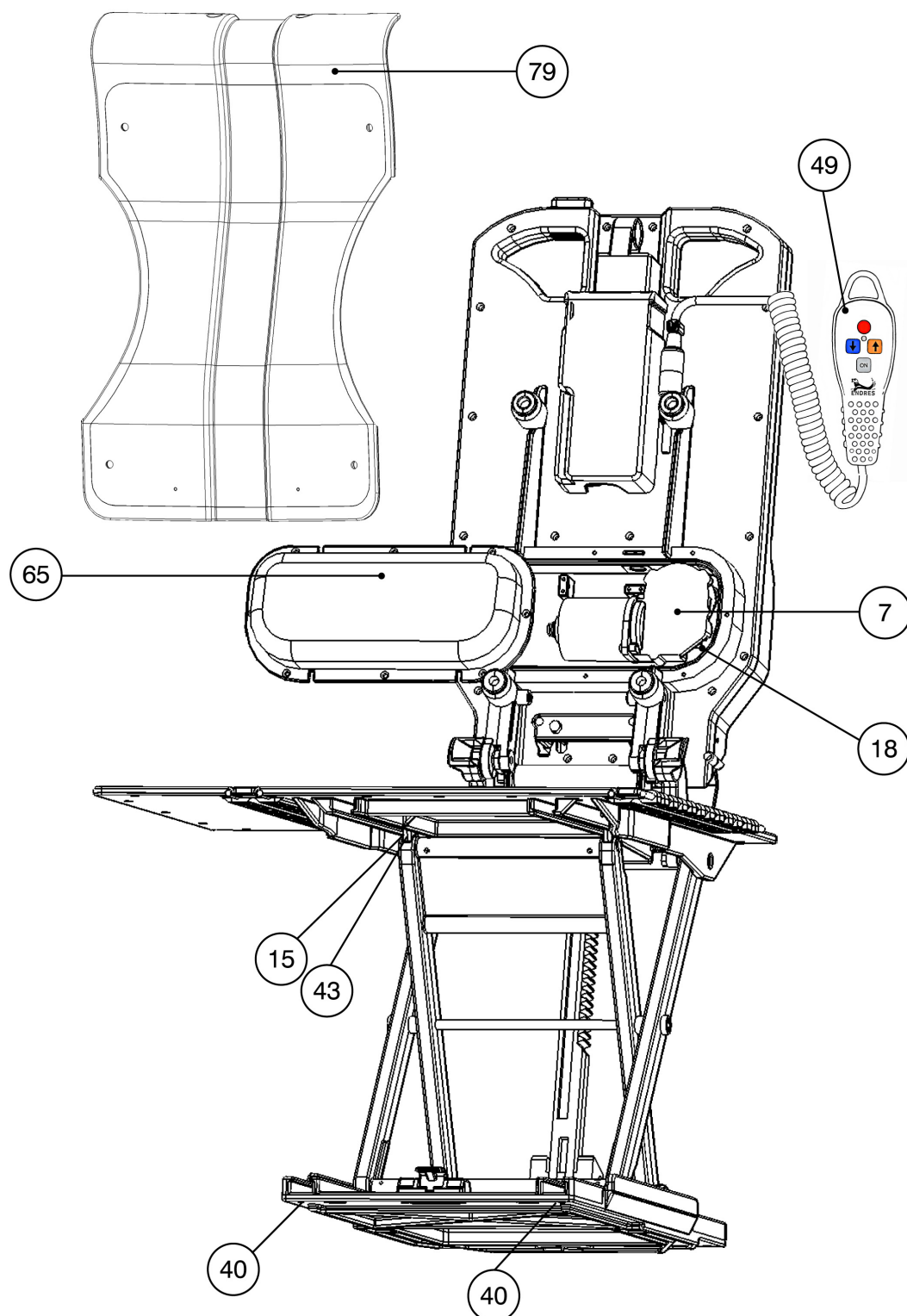

Picture 3 (P3)

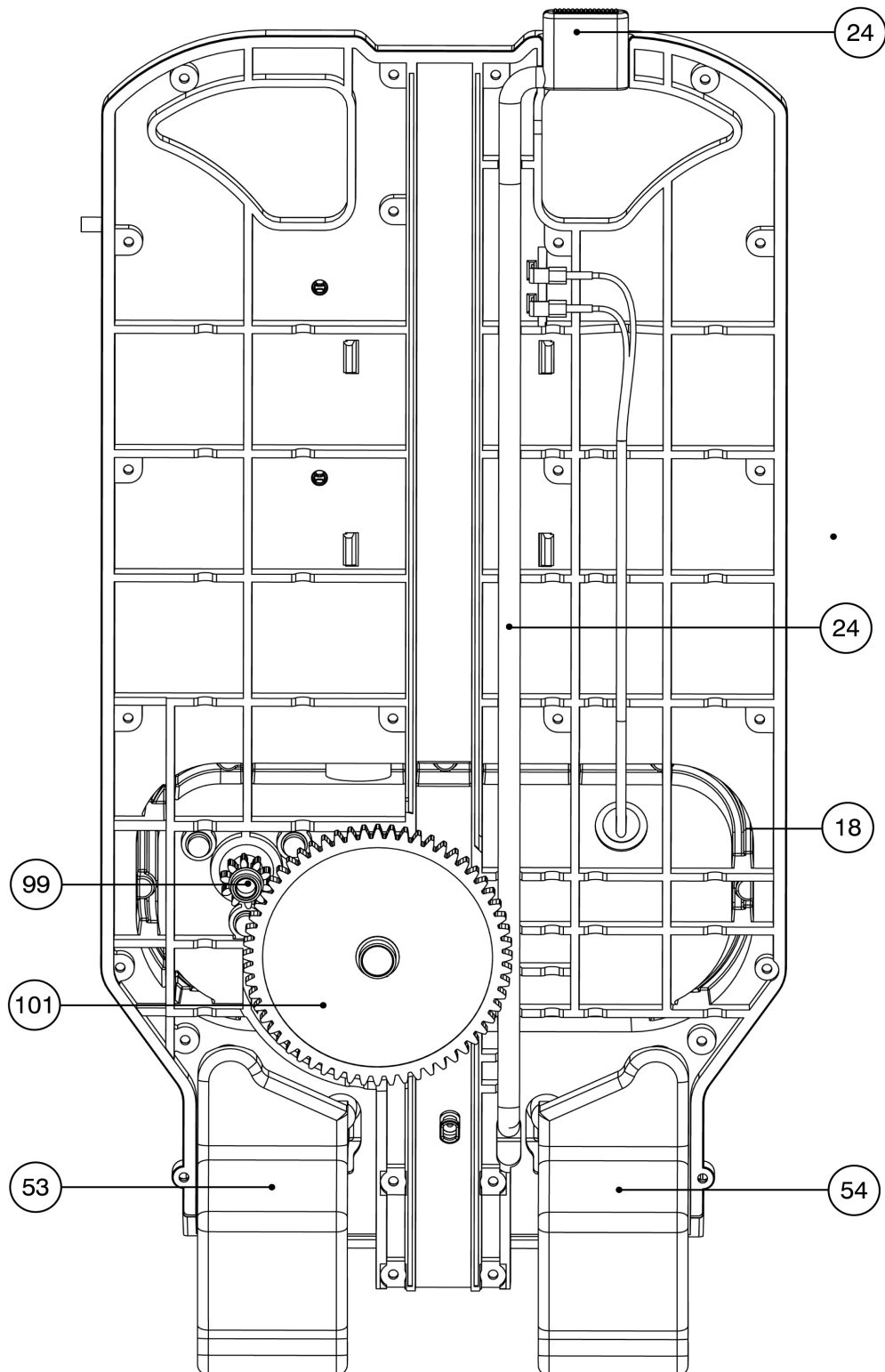

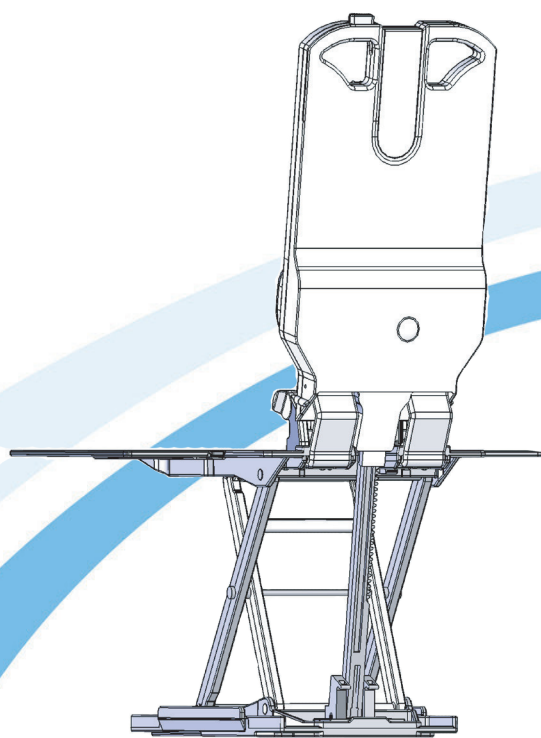

Stand July 2009

drive

Tel.: +49 (0) 7562 9724-0 • Fax: +49 (0) 7562 9724-25

Drive Medical GmbH & Co. KG • Leutkircher Straße 44D-88316 Isny/Allgäu • Germany • info@drivemedical.de • www.drivemedical.de